

iCube

Quick Bluetooth Pairing Guide



PHONAK life is on

Step 1

iPFG Successware

Install iPFG 2.0a or higher

Step 2



With NOAHlink:

- You can use the same Bluetooth adaptor for iCube (Ezurio BRBLU03 or Belkin F8T003). If you have already installed one of these adaptors proceed to Step 3.

Without NOAHlink:

- Phonak recommends the Ezurio USB Bluetooth adaptor BRBLU03 (available from Phonak).
- Insert the USB Bluetooth adaptor into a free USB port after installation of iPFG. No additional CD needs to be inserted.

Step 3



Start the Connection Wizard for iCube

Start iPFG and press the iCube status button which is located on the right-hand side of the iPFG screen.

Important information

Only use the pairing wizard via iPFG to pair iCube with your PC. Do not try to pair iCube under "Bluetooth environment" or change the already existing Bluetooth settings under Windows.

Step 4



Pairing iCube with your computer

Select "Use iCube via Bluetooth" and the pairing wizard will guide you through the process of pairing iCube with your computer.

Important information

If pairing fails, use the recommended Ezurio Bluetooth adaptor or contact the Phonak Fitting Software Support.