

# Phonak

## Fast Facts

### TV Connector

#### What is it?

The TV Connector is the easy plug and play streamer that directly connects compatible Phonak hearing aids to any TV or media player without the need of a body worn streamer. Thanks to AirStream™ technology, the TV Connector delivers top rated, streamed sound quality from any TV and stereo system.



#### Why use it?

The TV Connector offers clients

- A simple plug and play solution
- Dolby™ compatibility
- Easy broadcasting to multiple listeners within 15m (50ft) distance
- 3 activation modes (automatic, manual and silent)
- Manual controls with the new on/off and volume buttons

#### How to use it?

The TV Connector is plug and play right out of the box to any TV with optical (Toslink, digital) or 3.5 mm jack (analog) interfaces. Streaming begins automatically once the TV and TV Connector are turned on and the client is within a range (15 m/50 ft) from the TV Connector. The HCP can also

change the activation mode to suit the needs of the client. To connect multiple listeners, press the pairing button on the TV Connector. Clients within 1 m (3 ft) of the TV Connector will be automatically connected.

For more information on the TV Connector, go to [www.phonakpro.com/tvconnector](http://www.phonakpro.com/tvconnector) or contact your Phonak representative.



Manufactured under license from Dolby Laboratories. Dolby, Dolby Audio, and the double-D symbol are trademarks of Dolby Laboratories