


myPhonak App 6.0

Quick Guide



How to navigate through this guide

Follow the numbers to find the section you are interested in learning more about.

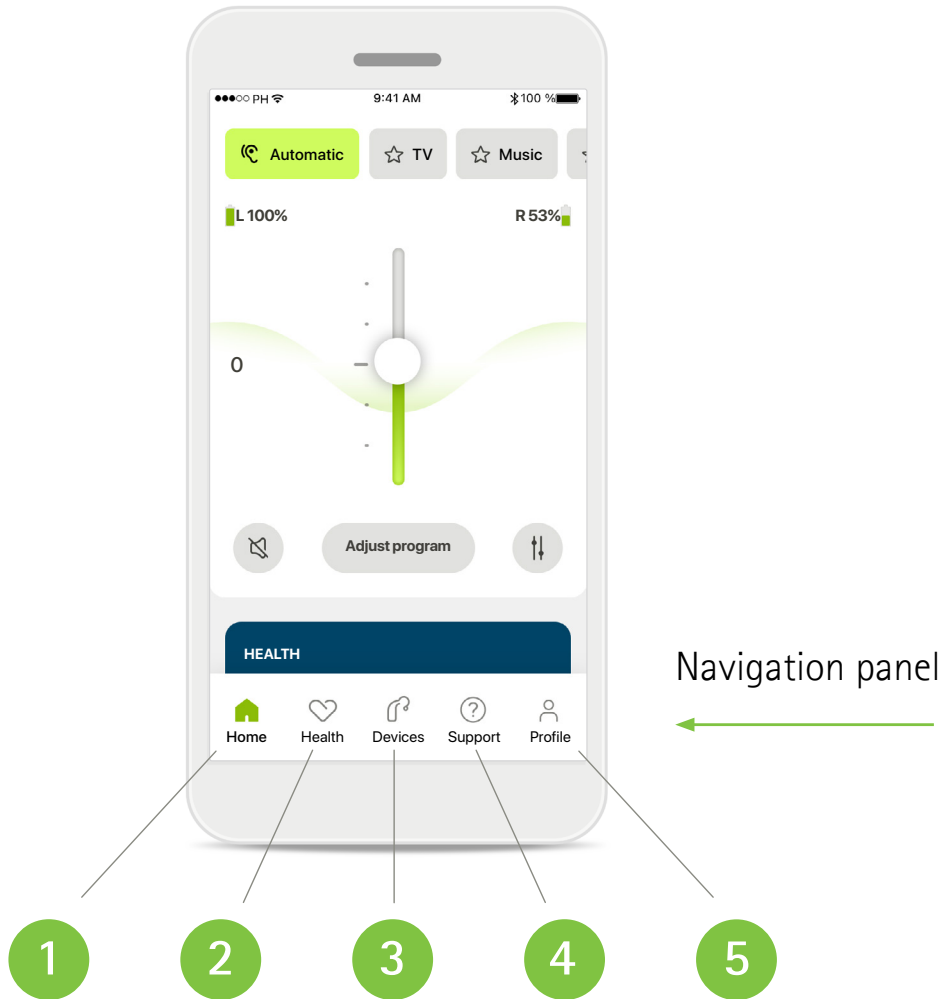



- 1** Home | Remote control, advanced sound settings
- 2** Health | Heart rate tracking*, daily steps**, wearing time**
- 3** Devices | Customize your hearing aid features
- 4** Support | Access remote support, FAQs, user guides
- 5** Profile | Access to profile information and preferences

* Audeo Fit Only

** Audeo Fit, Audeo Life, Audeo P-R, Audeo P-RT, Phonak Slim

1 Home

Offers personalization to the sound of your hearing aids

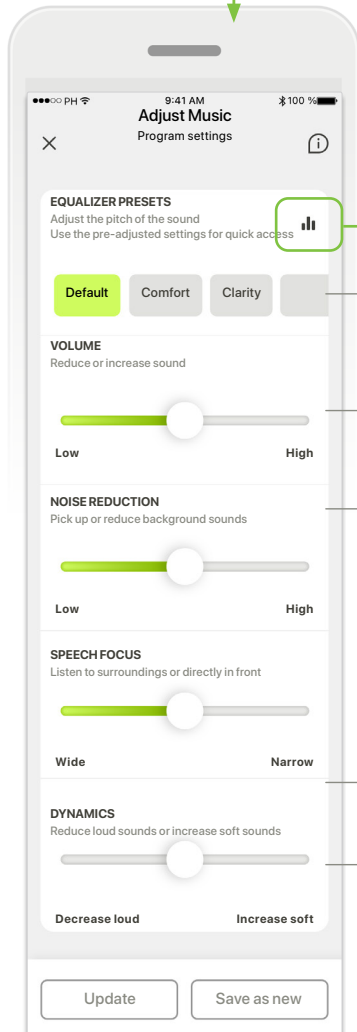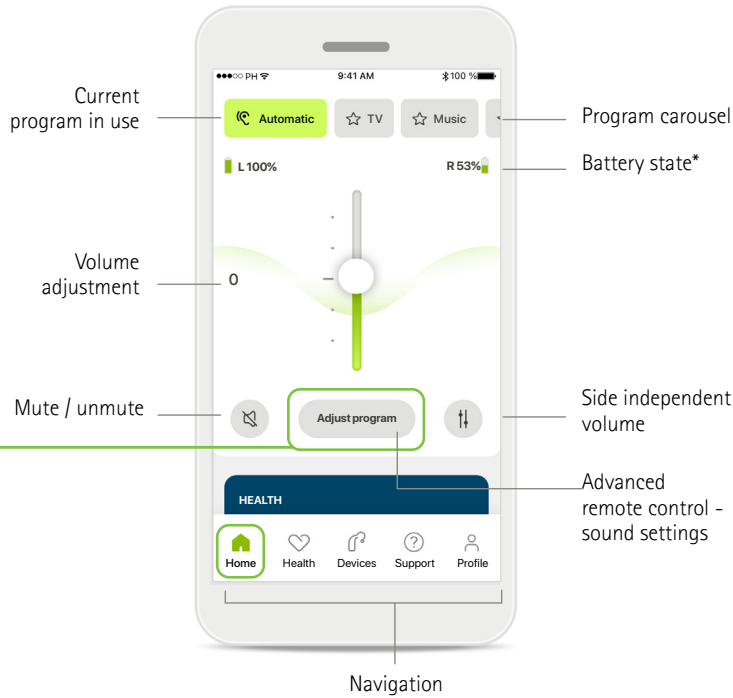

Equalizer
Adjust the pitch of the sound

Bass Middle Treble


Close

Equalizer allows you to modify the sound pitch by moving the sliders.

* For rechargeable hearing aids only


2 Health

- Some features in the Health section are available only with certain hearing aid models.
- To access the health features, you will need to create a myPhonak account with your email address.

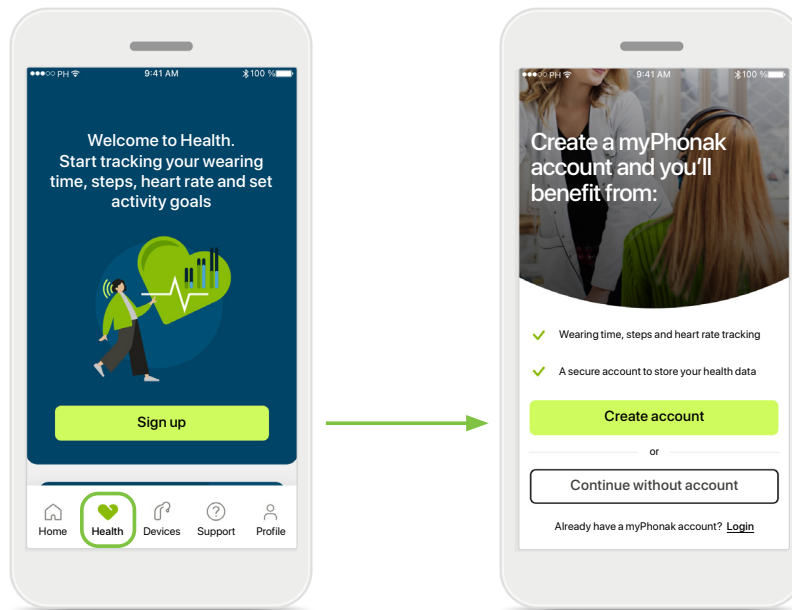


Wearing time

(Available with Audeo Fit, Audeo Life, Audeo P-R, Audeo P-RT, Phonak Slim)

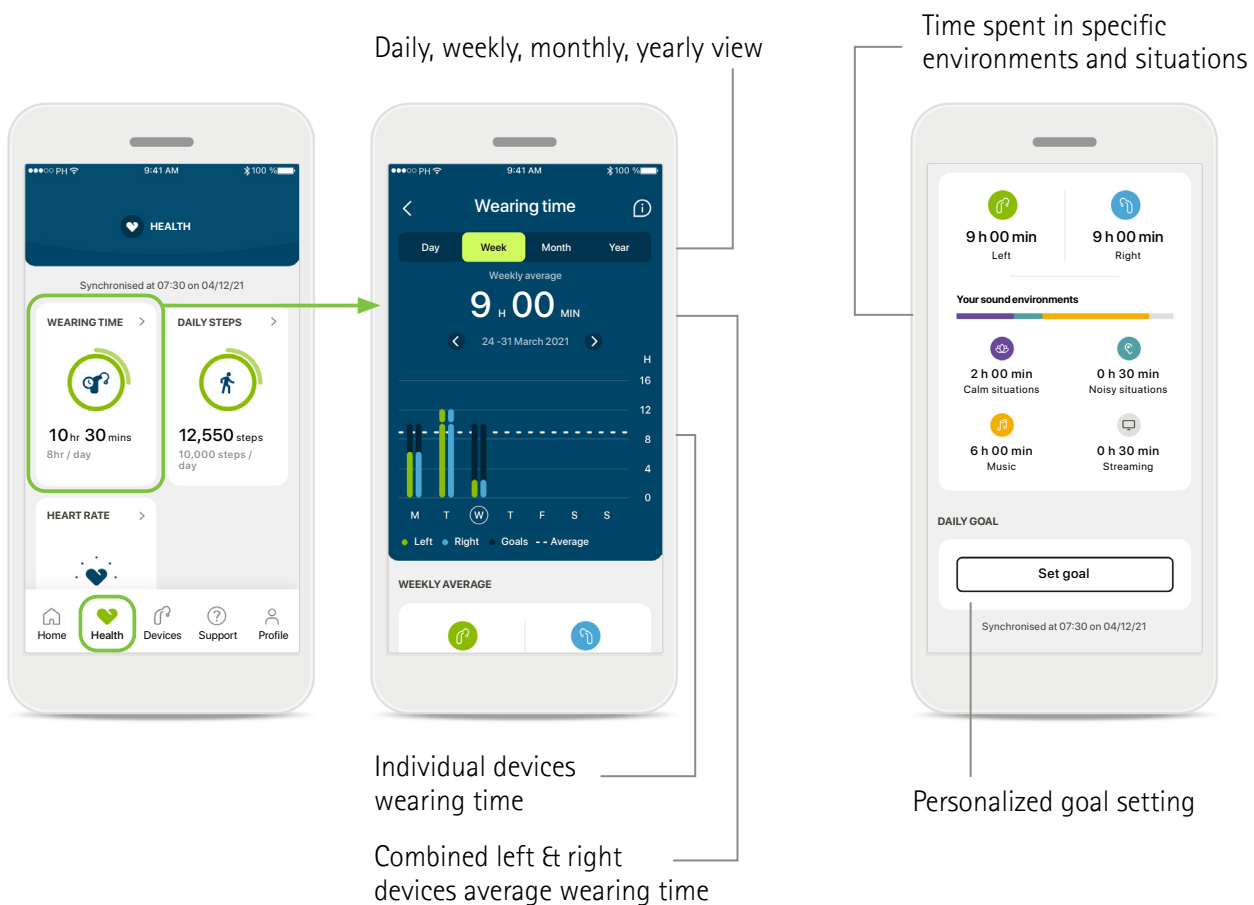


Steps

(Step tracker available with Audeo Fit, Audeo Life, Audeo P-R, Audeo P-RT, Phonak Slim; Distance walked or run only available with Audeo Fit)

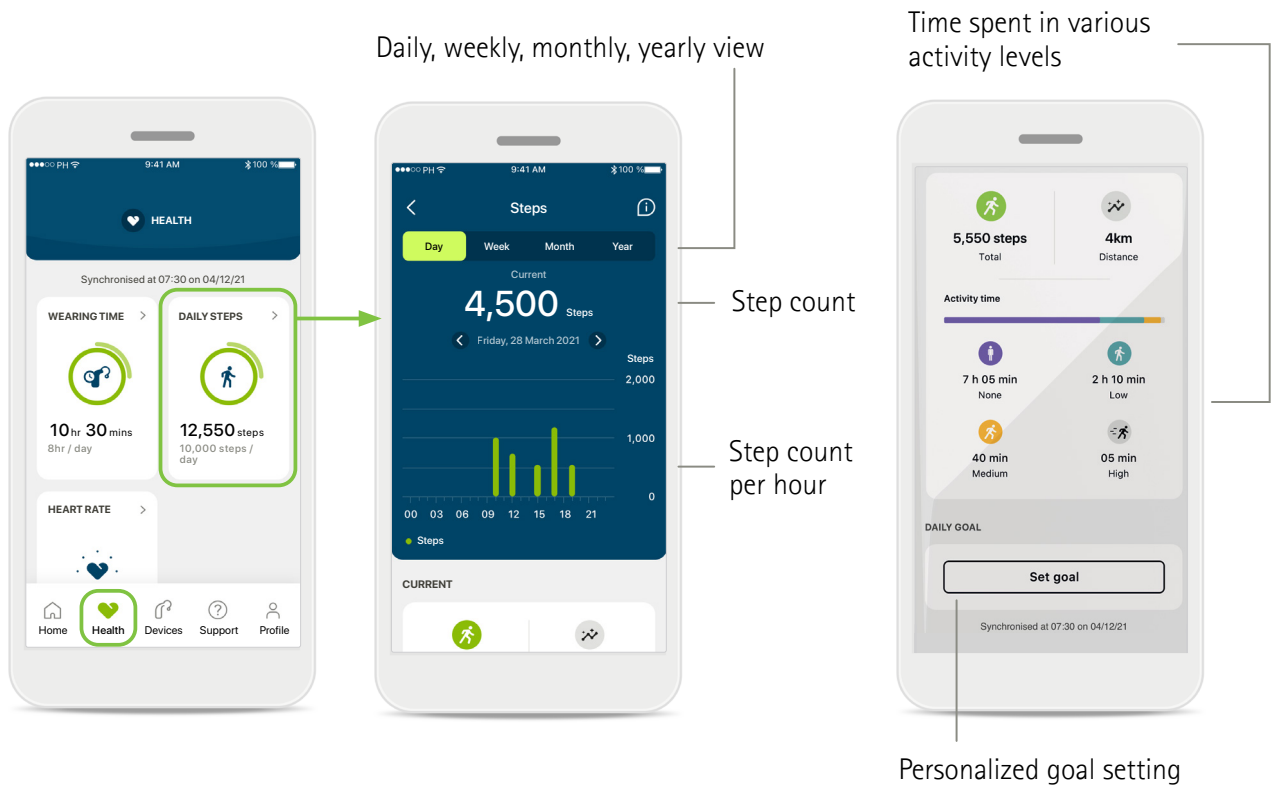

Heart Rate

(Available with Audeo Fit only)

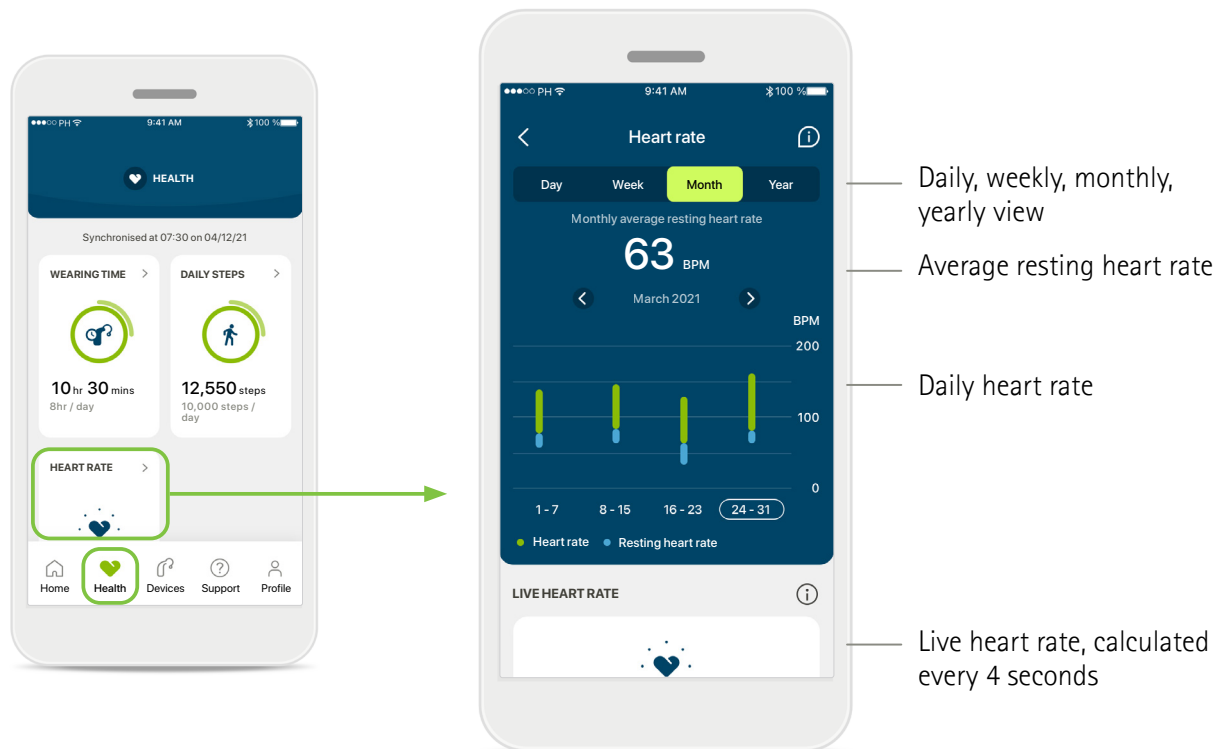

3 Devices

In the Devices section, you can access features related to your hearing aids.

- Select the subheadings for more options


The screenshot shows the 'DEVICE SETTINGS' menu with the following items:

- Bluetooth phone calls:** Change the sound quality of your calls. (Icon: Phone with Bluetooth symbol)
- Tap Control:** Change the sensitivity and functionality of your tap control. (Icon: Hand tapping) * Only available in select rechargeable models
- Auto-On:** Choose whether your aids automatically turn on when removed from the charger. (Icon: Power button with 'A')
- Cleaning Reminders:** Set up reminder. (Icon: Bell)
- YOUR DEVICE:** AUDÉO™ P90 Product information. (Icon: Hearing aid with question mark)


At the bottom, the navigation bar includes: Home, Health, **Devices** (highlighted), Support, and Profile.

4 Support

In the Support section, you can access user guides, how-to-videos, and **Remote Support**.

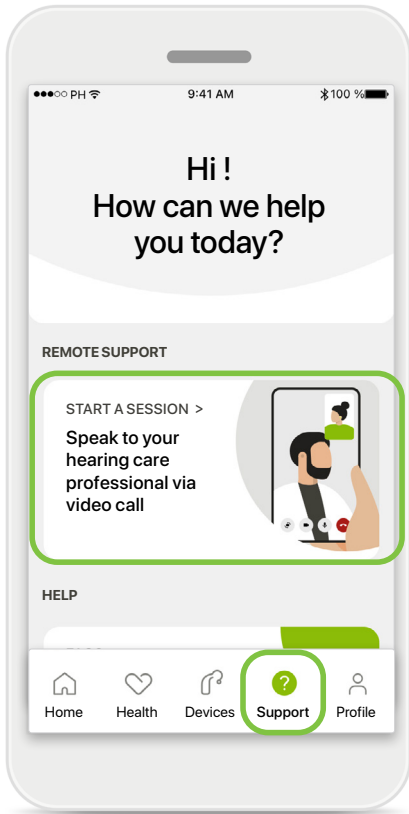

Click here to start a remote session with your hearing care professional at your agreed upon appointment time

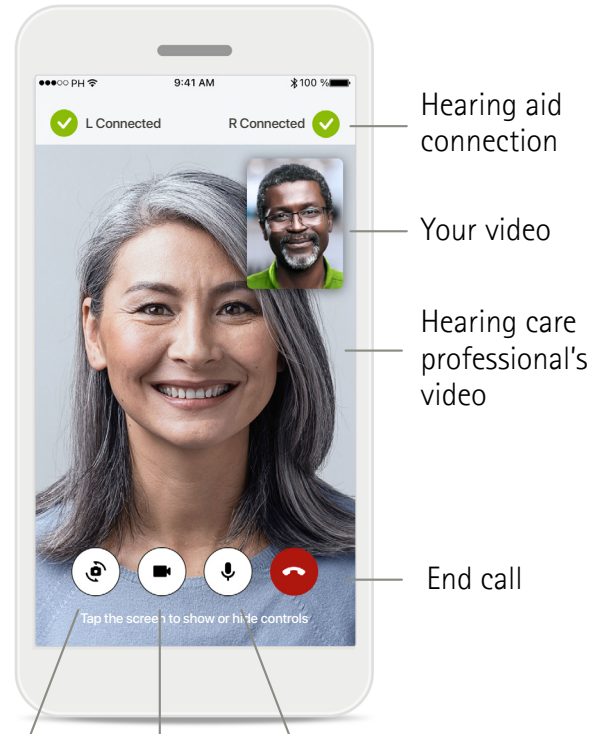

Hearing aid connection

Your video

Hearing care professional's video

End call

Flip camera view


Video on/off

Microphone on/off

5 Profile

Within the Profile section, you can:

- Edit and modify your personal information
- Delete your account
- Choose which data you want to share with Phonak

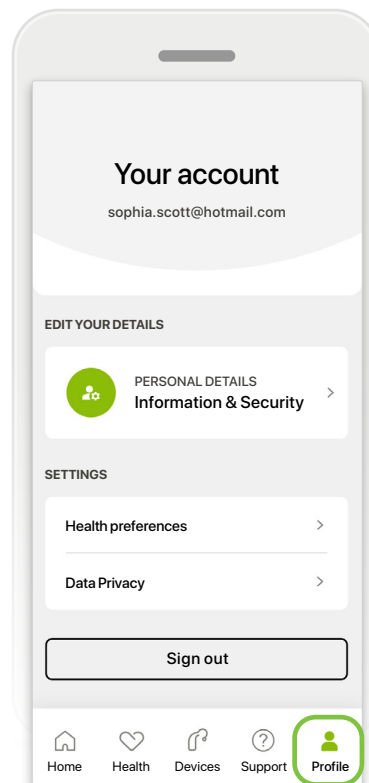

life is on

At Phonak, we believe that hearing well is essential to living life to the fullest. For more than 70 years, we have remained true to our mission by developing pioneering hearing solutions that change people's lives and help them to thrive socially and emotionally. Life is on.

www.phonak.com/ca

029-6697-02 / V1.00 / 2022-06 / Printed in 2022 © 2022 Sonova AG. All rights reserved.

sonova
HEAR THE WORLD



Manufacturer:
Sonova AG Laubisrütistrasse 28
CH-8712 Stäfa Switzerland www.phonak.com



CE
0459