

# Phonak Fast Facts



## Noahlink Wireless

### What is it?

Noahlink Wireless is the new industry-standard programming device for Bluetooth® Low Energy (BLE) enabled hearing aids. It is a HIMSA manufactured device that wirelessly transfers fitting data from the computer to BLE enabled hearing aids, such as Phonak Audéo™ B-Direct hearing aids, completely cable free.

### Why use it?

Noahlink Wireless simplifies the fitting process:

- It is a universal programming device eliminating the need for proprietary programming devices
- It provides the flexibility to adjust settings of BLE enabled wireless hearing aids, completely cable free
- Unique to Phonak, Audéo B-Direct can be easily identified. Press the push button on the hearing aid to highlight it in the selection of available devices.



### How is it used?

Noahlink Wireless is easy to use and to set up. Install Phonak Target 5.2 or higher and plug in the USB cable from the Noahlink Wireless into the fitting computer. Noahlink Wireless will be automatically installed and can be easily selected in the drop down menu of available fitting devices.

Open and close the battery door and click **[Connect]** to detect Phonak Audéo B-Direct. Devices in range will be shown. Press **[Play beep]** or the push button on the hearing aids to confirm which side/s you wish to assign to a client.

For more information on the Phonak Audéo B-Direct, go to [www.phonakpro.ca/audeo-b-direct](http://www.phonakpro.ca/audeo-b-direct) or contact your Phonak representative.

For more information on Noahlink Wireless, please refer to the FAQs on the HIMSA website <https://www.himsa.com/Products/NoahlinkWireless/ProductInformationSheets/tabid/4794/language/en-US/Default.aspx>

# Phonak Fast Facts



## TV Connector

### What is it?

The TV Connector is the easy plug and play streamer that directly connects Phonak Audéo™ B-Direct hearing aids to any TV or media player without the need of a body worn streamer. With AirStream™ Technology, the new Phonak proprietary 2.4 GHz streaming protocol, TV Connector offers a direct audio streaming from a broad range of TV and audio equipment in excellent stereo sound quality to Audéo B-Direct.

### Why use it?

#### Benefits to your clients

- No body worn streamer required
- Transforms Audéo B-Direct hearing aids into wireless stereo headsets
- Easy plug and play setup
- Provides high quality speech and music from any audio source.
- Environmental balance can be used to adjust the volume of the streamed signal or surrounding environmental sounds
- Unique advantage of connecting multiple listeners simultaneously to the same audio signal.

### How to use it?

The TV Connector is plug and play right out of the box to any TV with optical or 3.5mm interfaces. The hearing aids are automatically paired to the TV Connector. If pairing is not automatic, a one-time push on the TV Connector pairing button will connect the hearing aids within 1m (3ft). To connect multiple listeners, press the pairing button on the TV connector and clients within 1m (3ft) of the TV connector will be automatically paired. Turn the TV on and when the client is within a range (15m/49ft) from the TV Connector, streaming begins automatically.



For more information on the TV Connector, go to [www.phonakpro.ca/tvconnector](http://www.phonakpro.ca/tvconnector) or contact your Phonak representative.