

Phonak Mini Charger



What is it?

The Mini Charger is the small and compact charging option for Phonak Belong™ rechargeable hearing aids. Thanks to its size it takes up little space, making it ideal as a small charger to carry around or as a backup charger.



Phonak Mini Charger for Bolero B-PR

Why use it?

For clients who want a charger that is small, compact and more convenient to carry around, the Mini Charger is the ideal solution.

How to use it?

To charge the hearing aids simply click them into the charging slots of the Mini Charger and they automatically switch off. The green indicator light flashes slowly to indicate charging. Once the light stops flashing and remains solid the hearing aids are fully-charged and ready to use. Charging time for completely depleted hearing aids is 3 hours.




Phonak Mini Charger for Audéo B-R

For more information on the Phonak Bolero B-PR, Audéo B-R or the charging options, go to www.phonakpro.ca or contact your Phonak representative.

028-1700-02/2017-03/Printed in Canada © 2017 Sonova AG All rights reserved

Fast Facts Rechargeable

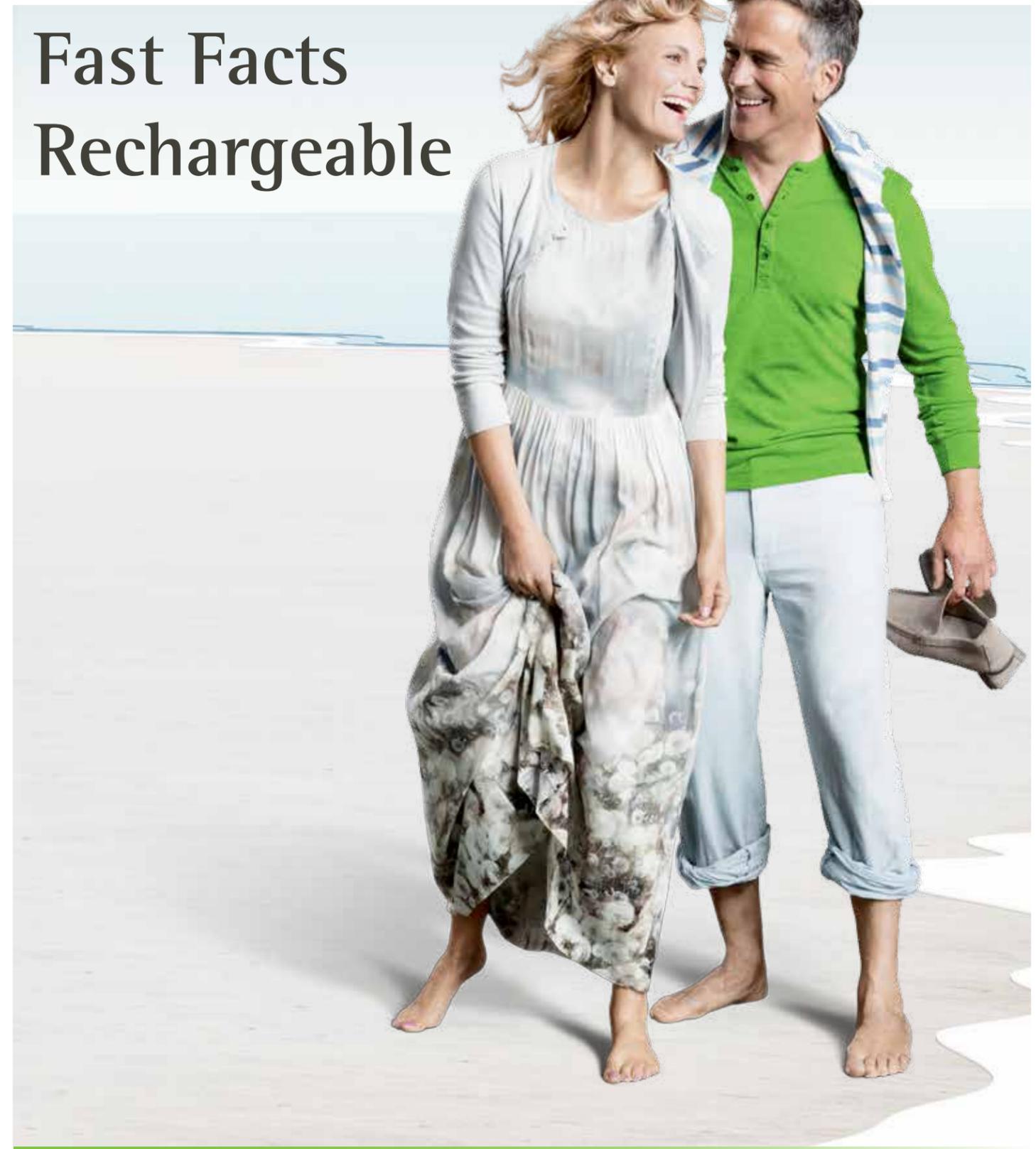

A Sonova brand

Phonak Rechargeable Hearing Instruments



What is it?

Phonak Bolero B-PR (BTE) and Audéo B-R (RIC) are fully featured hearing aids with a telecoil, volume control and a built-in rechargeable battery, eliminating the need to replace batteries. Both models offers 24 hours* of battery life on a single charge by using a lithium-ion (Li-ion) battery. Additionally, the new housing design offers the benefits of more durability with an IP rating of 68 (water resistant and dust tight).



Bolero B-PR



Audéo B-R

Why use it?

With rechargeable batteries, clients benefit by having hassle-free hearing without having to change batteries or carry extra batteries with them. Compared to other rechargeable technologies, Li-ion batteries have the additional benefit of reduced deterioration and degradation of the battery over time without affecting hearing aid performance. Furthermore removing the battery door has improved the durability of the hearing aid. You also benefit by having more counselling time which can be spent on topics not related to battery changing and maintenance.

How to use it?

Bolero B-PR and Audéo B-R are easy to use. Simply remove the hearing aids from the compatible charger and press the push button for 3 seconds until the light indicator briefly flashes green. The hearing aids are now switched on and ready for use. At the end of the day, simply place the hearing aids into the charger. The hearing aids will click into the slots and automatically switch off. The green light indicator flashes slowly to indicate charging. Once the light stops flashing and remains solid the hearing aids are fully charged and ready to for use. Charging time for a completely depleted hearing aid is 3 hours.

For more information on the Phonak Bolero B-PR, Audéo B-R or the charging options, go to www.phonakpro.ca or contact your Phonak representative.

*Expected results when fully charged, and up to 80 minutes wireless streaming time. Please refer to www.phonakpro.com/evidence

Phonak Charger Case and Power Pack



What is it?

The Phonak Charger Case is a charging option for Phonak Belong™ rechargeable hearing aids that also acts as a drying kit and a hardcase. An optional and removable Power Pack can be attached to the Charger Case for moments when there isn't an available external power source.

Why use it?

For clients who want a charger that has the benefits for charging, storing and drying their hearing aids at the same time, this is the ideal solution. By adding the Power Pack, there is the added benefit of charging without an external power source for up to 7days.

How to use it?

To charge the hearing aids, simply click them into the charging slots of the Charger Case and they automatically switch off. The green indicator light flashes slowly to indicate charging. Once the light stops flashing and remains solid the hearing aids are fully charged and ready to for use. Charging time for a completely depleted hearing aid is 3 hours. When there is no external power source available, the Power Pack can be locked to the bottom of the Charging Case. This will provide 7 full charges for a pair of Belong rechargeable hearing aids. To recharge the Power Pack, it does not have to be removed from the charger case. Just connect to an external power source and the hearing aids and power pack can be charged together.

For more information on the Phonak Bolero B-PR, Audéo B-R or the charging options, go to www.phonakpro.ca or contact your Phonak representative.



Phonak Charger Case Bolero B-PR

Phonak Charger Case Audéo B-R

Phonak Power Pack