

Phonak Fast Facts



Phonak Audéo™ B-R

What is it?

Phonak Audéo B-R is the first ever RIC hearing aid to feature a built-in lithium-ion (Li-ion) rechargeable battery that provides 24 hours* of hearing with one simple charge. This groundbreaking technology frees hearing aid wearers from the hassles of disposable batteries that are small and difficult to handle. Additionally, the new housing design offers the benefits of being water resistant and dust tight (IP68 rating).

Why use it?

With rechargeable batteries, clients benefit by having hassle-free hearing without having to change batteries or carry extra batteries with them. Using a Li-ion battery, there is the additional benefit of reduced deterioration and degradation of the battery over time without affecting hearing aid performance. Furthermore by removing the battery door, this has improved the durability of the hearing aid. Hearing care professionals also benefit by having more counselling time that can be spent on topics not relating to battery changing and maintenance.



How to use it?

Audéo B-R is easy to use. Simply remove the hearing aids from the charger and press the push button for 3 seconds until the indicator light briefly flashes green. The hearing aids are now switched on and ready for use. At the end of the day, simply place the hearing aids into the charger. The hearing aids will click into the slots and automatically switch off. The green indicator light flashes slowly to indicate charging. Once the light stops flashing and remains solid the hearing aids are fully charged and ready to for use. Charging time for completely depleted hearing aids is 3 hours.

For more information on the Phonak Audéo B-R rechargeable hearing aids and charging options, go to www.phonakpro.com/audéo-b or contact your Phonak representative.

*Expected results when fully charged, and up to 80 minutes wireless streaming time. Please refer to www.phonakpro.com/evidence.