

Phonak Fast Facts



Phonak Charger Case and Power Pack

What is it?

The Phonak Charger Case is a charging option for Phonak Belong™ rechargeable hearing aids that also acts as a drying kit and a hardcase. An optional and removable Power Pack can be attached to the Charger Case for moments when there isn't an available external power source.

Why use it?

For clients who want a charger that has the benefits for charging, storing and drying their hearing aids at the same time, this is the ideal solution. By adding the Power Pack, there is the added benefit of charging without an external power source for up to 7 days.

How to use it?

To charge the hearing aids, simply click them into the charging slots of the Charger Case and they automatically switch off. The green indicator light flashes slowly to indicate charging. Once the light stops flashing and remains solid the hearing aids are fully charged and ready for use. Charging time for a completely depleted hearing aid is 3 hours. When there is no external power source available, the Power Pack can be locked to the bottom of the Charging Case. This will provide 7 full charges for a pair of Belong rechargeable hearing aids. To recharge the Power Pack, it does not have to be removed from the charger case. Just connect to an external power source and the hearing aids and power pack can be charged together.



Phonak Charger Case



Phonak Charger case with the Power Pack attached

For more information on the Phonak Beyond rechargeable hearing aids and charging options, go to www.phonakpro.com/audeo-b or contact your Phonak representative.