

Phonak Mini Charger

What is it?

The Mini Charger is the small and compact charging option for Phonak Belong™ rechargeable hearing aids. Thanks to its size it takes up little space, making it ideal as a small charger to carry around or as a backup charger.

Why use it?

For clients who want a charger that is small, compact and more convenient to carry around, the Mini Charger is the ideal solution.



How to use it?

To charge the hearing aids simply click them into the charging slots of the Mini Charger and they automatically switch off. The green indicator light flashes slowly to indicate charging. Once the light stops flashing and remains solid the hearing aids are fully charged and ready to use. Charging time for completely depleted hearing aids is 3 hours.

For more information on the Phonak Belong rechargeable hearing aids and charging options, go to www.phonakpro.com/audeo-b or contact your Phonak representative.

