


Phonak Marvel™ Wireless accessories

Product information

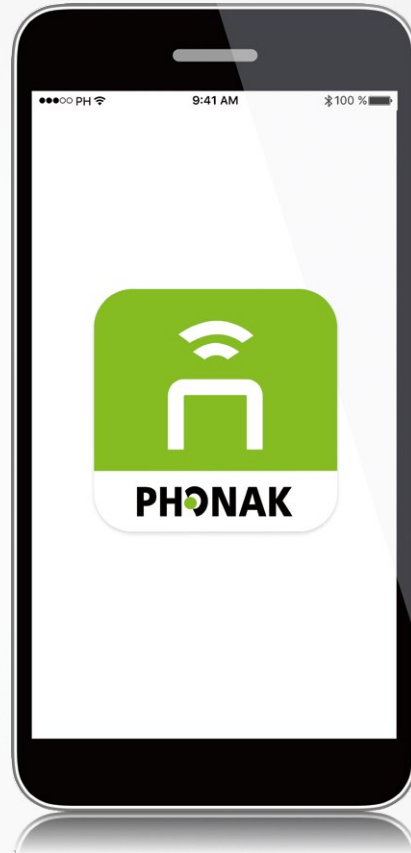

Phonak TV Connector is an easy to use plug and play solution for Phonak Marvel hearing aids. Featuring proprietary Phonak AirStream™ technology, it delivers top rated sound quality streaming from any TV and stereo system. Additionally, with direct connectivity to iOS and Android phones, the user friendly Phonak Remote app allows a user to get even more out of their Marvel hearing aid.

TV Connector – The easy to use streaming device for TV and music



The **TV Connector** is the plug'n play interface to TV and other audio sources, supporting the direct connectivity to Phonak Marvel hearing aids. It's designed to simplify the setup and usage and offers excellent stereo sound quality using the AirStream™ technology. The TV Connector can stream audio signals over a distance of up to 15 meters to an unlimited amount of listeners connected at the same time.


Feature overview

- Streaming to both hearing aids in stereo sound
- AirStream technology for better sound quality, lowest streaming latency and easier connection method
- "One-click" to connect all compatible hearing aids within range without a gesture on the hearing aid
- Simultaneous streaming to an unlimited amount of users at the same time
- For more discreetness, it can be mounted behind the TV

Compatibility

- The TV Connector is compatible with all Phonak direct connectivity wireless hearing aids.

How to use

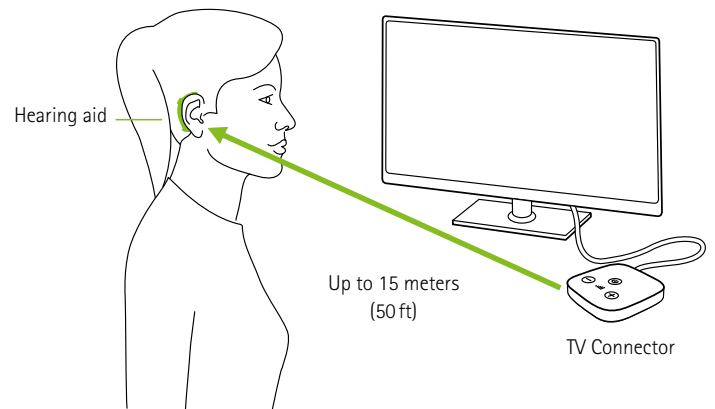

For videos on how to use and setup the TV Connector, please visit www.phonakpro.com/tvconnector

Technical facts



- 1 Connect button
- 2 Indicator light (LED)
- 3 microUSB power socket
- 4 Audio socket for optical (Toslink) or analog (3.5mm jack) cable
- 5 Power button
- 6 Volume up and down buttons (calibration)

Small housing size (L: 63 mm x W: 63 mm x H:12 mm)

Phonak Remote – The simplified remote control app




The **Phonak Remote** app is designed to improve the usability for more convenience when using a remote control. It is an easy control to your hearing aid functions, visualized on the smartphone without the need for an intermediate, body-worn device.


Feature overview

- Volume adjustment of the hearing aids for both sides or each side individually
- Program list with direct selection of available hearing aid programs
- Possibility to customize the name of the available hearing aid programs in the app
- Selection of available audio sources connected to the hearing aids
- Easy adjustment of the environmental balance functionality of the hearing aid during a stream from an audio source
- Separate adjustment of the Tinnitus Noiser intensity when available on the hearing aid
- Check the average wearing time of the connected hearing aids on the app
- Battery State of charge (rechargeable hearing aids)
- Configuration of Auto-On when out of charger (rechargeable hearing aids)


For videos on how to use and setup the Phonak Remote app on your phone, please visit www.phonakpro.com/remoteapp

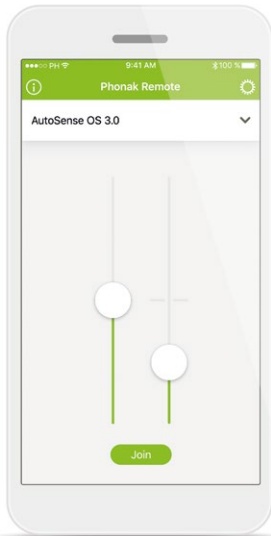


Volume adjustment for each side

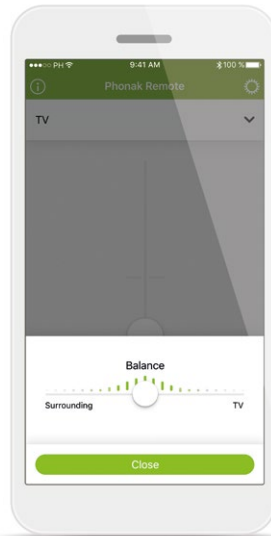

Environmental balancing of an audio stream

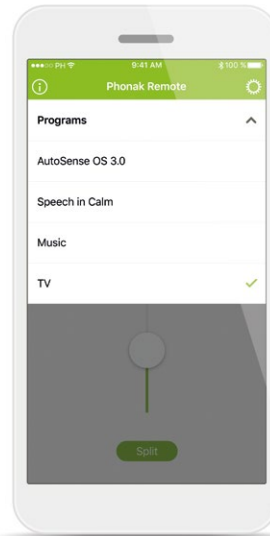

Program selection and customization

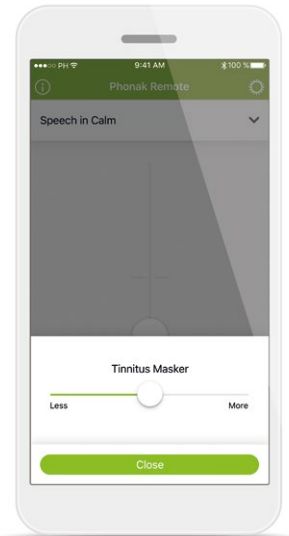

Tinnitus Noiser intensity

Compatibility

- The Phonak Remote app is compatible with Phonak hearing aids with Bluetooth® connectivity.
- Phonak Remote can be used on Phones with Bluetooth® Low-Energy (BT-LE) capability and is compatible with iPhone 5s and newer running iOS Version 10.2 or newer.
- Phonak Remote can be used on Google Mobile Services (GMS) certified Android devices supporting Bluetooth 4.2 and Android OS 6.0 and newer.

Apple, the Apple logo, iPhone, and iOS are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

Android, Google Play, and the Google Play logo are trademarks of Google Inc.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc.

Life is on

At Phonak, we believe that hearing well is essential to living life to the fullest. For more than 70 years, we have remained true to our mission by developing pioneering hearing solutions that change people's lives to thrive socially and emotionally. Life is on.

www.phonak.com