

# Phonak CROS B

## Product information



Phonak CROS B is a product for clients with an unaidable hearing loss in one ear. It is worn on the unaidable ear in combination with a Belong™ hearing aid on the better ear. CROS B transfers the microphone input signal to the hearing aid. The hearing aid receives the signal and plays it into the better ear to make sounds coming from the unaidable side audible. CROS B has no acoustical output.

CROS B is compatible to hearing aids in all performance classes using the Phonak Belong platform. It can be fitted with Phonak Target 5.0 and later versions.

# Product description



CROS B-R



CROS B-312



CROS B-13



CROS B-312 Custom



CROS B-13 Custom

|                          | CROS B-R  | CROS B-312                                      | CROS B-13                                       | CROS B-312 Custom      | CROS B-13 Custom       |
|--------------------------|---|---|---|------------------------|------------------------|
| Battery size             |   | 312   | 13  | 312                    | 13                     |
| Working current (mA)     |   | 2.6   | 2.6   | 2.1                    | 2.1                    |
| Push button              | •   | •   | •   | •                      | •                      |
| Volume control           |   |   | •   | Optional               | Optional               |
| Tamperproof battery door |   |   | •   |                        |                        |
| Nano coating             | •   | •   | •   | •                      | •                      |
| IP rating                | IP68 <sup>1</sup>                               | IP68 <sup>1</sup>                               | IP68 <sup>1</sup>                               |                        |                        |
| Dimensions (L x W x D)   | 29.7 x 12.9 x 7.9 mm<br>(1.17 x 0.51 x 0.31 in) | 26.6 x 10.8 x 6.9 mm<br>(1.05 x 0.43 x 0.27 in) | 31.6 x 14.4 x 8.4 mm<br>(1.24 x 0.57 x 0.33 in) |                        |                        |
| Weight                   | 2.4g (0.09 oz)                                  | 1.3g (0.046 oz)                                 | 2.15g (0.076 oz)                                |                        |                        |
| Compatibility            | Audéo B-R                                       | Belong RIC/BTE                                  | Belong RIC/BTE                                  | Belong custom products | Belong custom products |

<sup>1</sup> IP68 indicates that the hearing aid is water resistant and dust protected. It survived immersion in 1m of water for 60 minutes and 8 hours in a dust chamber as per the IEC60529 standard. No traces of dust were evident within the housing.

## Colors

|  |                             |                               |                       |                           |                          |                                |                              |                               |                    |
|--|-----------------------------|-------------------------------|-----------------------|---------------------------|--------------------------|--------------------------------|------------------------------|-------------------------------|--------------------|
| <b>CROS B-312<br/>CROS B-13<br/>CROS B-R</b> | <b>P1</b><br>Sand Beige     | <b>P3</b><br>Sandalwood       | <b>P4</b><br>Chestnut | <b>P5</b><br>Champagne    | <b>P6</b><br>Silver Gray | <b>P7</b><br>Graphite Gray     | <b>P8</b><br>Velvet Black    | <b>T7</b><br>White            | <b>01</b><br>Beige |
| <b>CROS B-13</b>                             | <b>Q2</b><br>Electric Green | <b>Q3</b><br>Caribbean Pirate | <b>T3</b><br>Chestnut | <b>T7</b><br>Alpine White | <b>M6</b><br>Lava Red    | <b>M7</b><br>Blue Ocean        | <b>M8</b><br>Majesty Purple  |                               |                    |
| <b>Ear hook<br/>(CROS B-13 only)</b>         | <b>R1</b><br>Green          | <b>R2</b><br>Yellow           | <b>R3</b><br>Pink     | <b>06</b><br>Blue         | <b>13</b><br>Tranparent  | <b>Q7</b><br>Purple            | <b>R4</b><br>Orange          |                               |                    |
| <b>Custom shell</b>                          | <b>14</b><br>Tan            | <b>22</b><br>Cocoa            | <b>26</b><br>Pink     | <b>28</b><br>Brown        | <b>19</b><br>White       | <b>21</b><br>Clear Transparent | <b>36</b><br>Red Transparent | <b>37</b><br>Blue Transparent |                    |
| <b>Faceplate</b>                             | <b>14</b><br>Tan            | <b>22</b><br>Cocoa            | <b>26</b><br>Pink     | <b>28</b><br>Brown        |                          |                                |                              |                               |                    |

## Retention options

### CROS DemoHook

- Hook to hold CROS on the ear during fitting and demonstration
- For demo purposes only
- Included with any Phonak CROS device

### CROS Tube

- Standard SlimTube for CROS devices
- Available in left/right and four different lengths (0, 1, 2, 3)
- Dome needs to be attached. Available as open, closed and power dome in three sizes (S, M, L)

### CROS Hook

- Earhook to attach a custom earmold with standard tubing to CROS -13 models
- Only available for the CROS -13 models
- Included with any CROS -13 device or can be ordered separatly (order number: 048-0261)

### CROS Tip

- Custom made earpiece for CROS devices
- Available in left/right and four different lengths (0, 1, 2, 3)

# Fitting

|            |   |
|------------|---|
| Software   | Phonak Target 5.0 or higher, 5.2 for CROS B Custom models, 5.3.3 for CROS B-R |
| Interfaces | iCube II, NOAHLink, HI-PRO, HI-PRO2   |

When using iCube II, Phonak CROS B is active during fitting.  
Your clients can instantly experience the benefits of the CROS solution and provide feedback during the fitting session.

## Wireless Communication Portfolio

All wireless accessories can be connected and used with the hearing aid on the better ear.  
If a streaming signal is received by the hearing aid, the streaming signal from CROS is automatically put on hold.  
The connection to CROS will be automatically re-established when streaming from the accessory is stopped.

## Key features

### Demonstration of CROS during fitting

When using iCube II, CROS B is active during fitting. This leads to a quicker and more precise fitting process since the patient can give instant feedback on the balance between the devices and the sound quality.

### StereoZoom

Zooms in on a single voice in a diffuse, noisy environment improving speech intelligibility by up to 60%<sup>1</sup>. StereoZoom is available for the 90 (manual program and automatic) and the 70 (manual program) performance levels.

<sup>1</sup> StereoZoom: Improved speech understanding even with open fittings.  
Phonak Field Study News, April 2011

### CROS volume control

Increasing the comfort in noise, the CROS volume control allows the client to individually adjust the volume of the CROS signal.

### AutoSense OS

AutoSense OS automatically senses your client's exact listening environment in real-time. It draws from multiple features, while accurately blending them to create over 200 distinct settings.

## CROS B performance

CROS B also offers the Speech in Loud Noise program with StereoZoom. It is available in combination B90 and B70 performance level hearing aids. For the lower performance levels, CROS B provides Real Ear Sound and omnidirectional microphone modes.

On the better ear, CROS B clients fully benefit from all Belong features like AutoSense OS. It provides seamless transitions and optimum settings for different listening situation.

| Hearing aid performance level | Available microphone modes for CROS B |                   |                |                 |
|-------------------------------|---------------------------------------|-------------------|----------------|-----------------|
|                               | Automatic StereoZoom                  | Manual StereoZoom | Real Ear Sound | Omnidirectional |
| Premium (B90)                 | •                                     | •                 | •              | •               |
| Advanced (B70)                |                                       | •                 | •              | •               |
| Standard (B50)                |                                       |                   | •              | •               |
| Essential (B30)               |                                       |                   |                | •               |

---

# Life is on

At Phonak, we believe that hearing well is essential to living life to the fullest. For more than 70 years, we have remained true to our mission by developing pioneering hearing solutions that change people's lives to thrive socially and emotionally. Life is on.

[www.phonakpro.com](http://www.phonakpro.com)