

Lyric₃ programming wand

User Guide for professionals



PHONAK
life is on

Welcome


Introducing the new Lyric3 programming wand from Phonak.

The Lyric3 programming wand is designed specifically for use by you, the hearing care professional, and enables the wireless transfer of fitting data from your PC to your client's Lyric3 hearing instruments.

Please read this manual carefully to benefit from all the features of your Lyric3 programming wand.

Phonak – life is on

Description

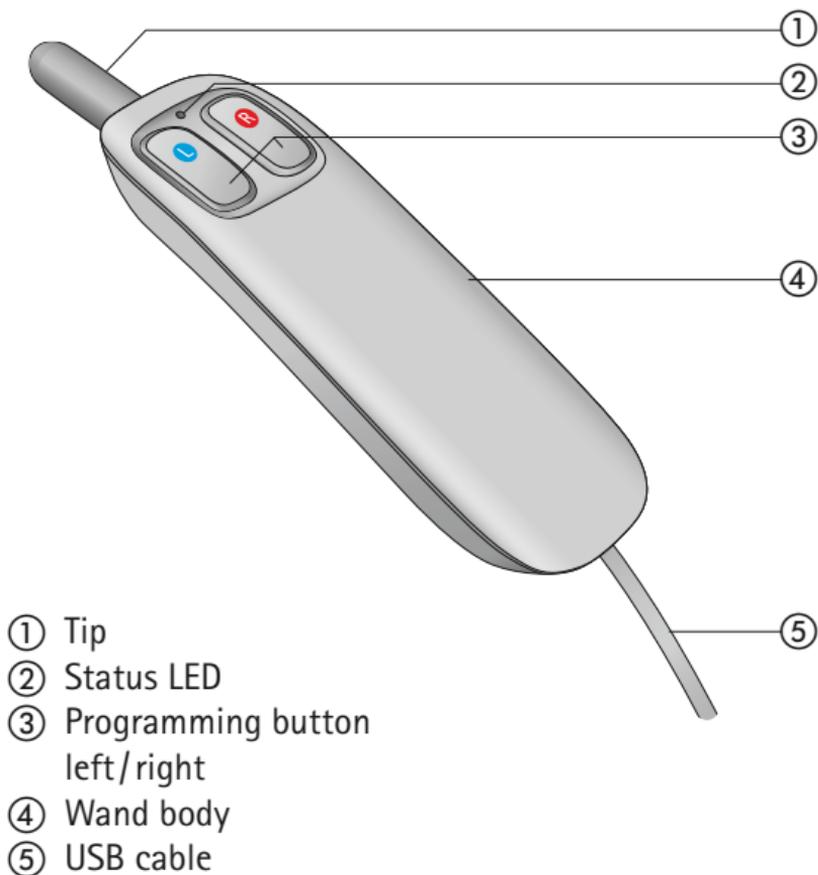


Getting started

To connect the Lyric3 programming wand, plug the USB-plug into an available USB-port on your fitting computer. There is no driver installation required for the Lyric3 programming wand.

After successful set-up, the status LED on the wand will turn green to indicate that the wand is ready for use.

Programming

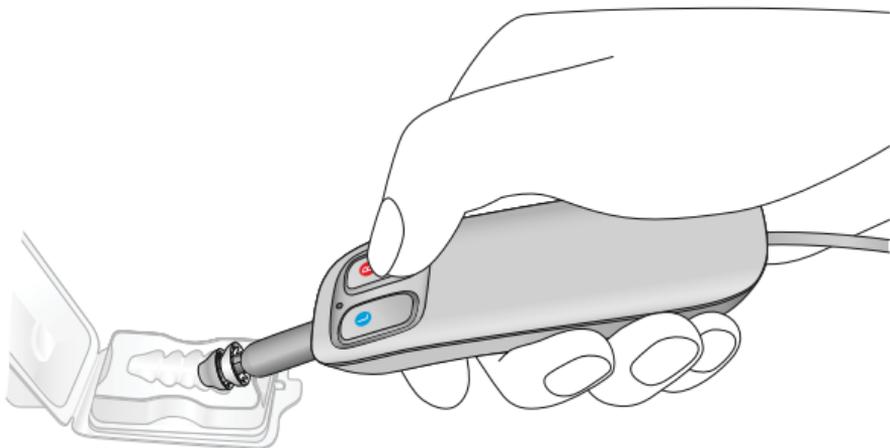

To program, hold the tip of the wand close to the lateral end of the Lyric3 hearing aid and press the programming button according to the ear you want to fit. The status LED will flash green during programming.

The status LED turns back to solid green after 3 seconds. The wand is then ready for the next programming procedure.

Lyric3 will play a double beep to indicate that it has been programmed. When Lyric3 is programmed for the first time, it will switch from "Ship Mode" to ON when successfully programmed. This can be verified by listening for feedback or using a listening stethoscope.

If programming is performed in the ear, ask the patient to pay attention and confirm the beeps to make sure the programming was successful.

Further indicators

If an error occurs during programming, the status LED will flash orange for 3 seconds before turning back to green. In that case repeat the programming procedure.

If any other malfunctions occur with the wand, the status LED will turn red. In that case, reboot the wand by disconnecting and reconnecting it to the USB-port.

The status LED will flash red when the firmware of the wand is updated. Wait until it turns back to green before programming. Do not disconnect the wand during the firmware update.

Care and maintenance

To clean the programming tip of the wand use a disinfectant wipe if necessary. To clean the wand body, use a mild detergent. The wand must not be dipped in water or other detergents.

Compliance information

Hereby Phonak AG declares that this Phonak product is in compliance with the essential requirements of the Medical Devices Directive 93/42/EEC. The full text of the Declaration of Conformity can be obtained from the manufacturer or the local Phonak representative whose address can be taken from the list on www.phonak.com (worldwide locations).

Important safety information

Please read the information on the following pages before using your Lyric3 programming wand.

- ⚠ The intended use of the Lyric3 programming wand is to program Lyric3 hearing aids with fitting parameters.
- ⚠ Changes or modifications to the Lyric3 programming wand are not permitted. Such changes may damage the Lyric3 programming wand. Phonak is not responsible for situations resulting from a modified device.
- ⚠ The Lyric3 programming wand contains a magnet. If a medical device like a pacemaker is used, read the specific user guide to clarify possible impact of magnetic fields to the medical device. Keep a distance of at least 15 cm (6 inches) between the Lyric3 programming wand and the medical device.

-
- ① Never immerse the Lyric3 programming wand in water. Protect it from excessive moisture as it contains sensitive electronic parts.
 - ① Protect your Lyric3 programming wand from heat (never leave near window or in a car). Never use a microwave or other heating devices to dry the Lyric3 programming wand.
 - ① Do not drop the Lyric3 programming wand! Dropping onto a hard surface can damage your Lyric3 programming wand.
 - ① Do not use the Lyric3 programming wand in areas where electronic equipment is prohibited.

Information and description of symbols



With the CE symbol, Phonak AG confirms that this Phonak product meets the requirements of the Medical Devices Directive 93/42/EEC.



This symbol indicates that the products described in these user instructions adhere to the requirements for an application part of Type B of EN 60601-1. The surface of the Lyric3 programming wand is specified as an applied part of Type B.



This symbol indicates that it is important for the user to read and take into account the relevant information in this user guides.



EMC and radio communications compliance label Australia.



This symbol indicates that it is important for the user to pay attention to the relevant warning notices in these user guides.



Important information for handling and product safety.

Operating conditions

The product is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in these user guides.



Temperature: -20° to $+50^{\circ}$ Celsius (-4° to $+122^{\circ}$ Fahrenheit).



Keep dry



Humidity transportation: up to 90% non-condensing).

Humidity storage: 0% to 70%, if not in use.



Atmospheric pressure: 200 hPa to 1500 hPa.



The symbol with the crossed-out garbage bin is to make you aware that the Lyric3 programming wand may not be thrown away as normal household waste. Please return it to your group company. Proper disposal protects the environment and health.

**Manufacturer:**

Phonak AG
Laubisrütistrasse 28
CH-8712 Stäfa
Switzerland

US Representative:

Phonak LLC
4520 Weaver Parkway
US-Warrenville, IL 60555-3927
USA



www.lyrichearing.com



www.lyrichearing.co.uk



www.lyrichearing.co.au

www.phonak.com

