

Naída CI Connect Fitting Guide

Hardware and software compatibility

The following components are required in order to use Naída CI Connect.

Naída CI Q90

Only the Naída CI Q90 is compatible with Naída CI Connect.

A simple software upgrade in SoundWave™ 3.2 Fitting Software is required by your hearing care professional for the Naída CI in order to unlock the Naída CI Connect functionality.

PowerCel™ 170 battery

The Naída CI Connect can be attached to the PowerCel 170 battery.



Configuration of Naída CI Q90

Default program

Naída CI Connect is compatible with any Naída CI Q90 program via the RogerReady functionality. This ensures that the Naída CI Connect will automatically switch into streaming mode any time a streaming signal is received. The sound processor will automatically revert back to the full microphone 20 seconds after streaming has ended.

AUX only program

Should the patient wish to hear no environmental sound from the Naída CI Q90 microphones, then a manual Connect/Roger program should be added with an AUX only input source. Accessing this program requires a manual program change on the Naída CI Q90.