

Phonak Sky B is the newest and most comprehensive portfolio exclusively designed for children of all ages with mild to profound hearing loss.

Powered by the innovative technology of the Phonak Belong platform, Sky B offers six models in up to four performance levels, including a rechargeable BTE with the full Sky feature set, a telecoil and volume control. It features the world's only*, evidence based pediatric operating system, AutoSense Sky OS, the adaptive frequency lowering technology from Phonak, SoundRecover2, and an improved Roger and directional setting. All models are water and dust resistant with an IP68¹ rating, making them ideal for children's active lifestyles. The indicator light and tamperproof housings provide parents and caregivers additional confidence and security. Available in a variety of vibrant Mix & Match colors, Sky B allows children to fully express their personalities.

For more information visit www.phonakpro.com

* valid February 2018




Product description


					1	Š	
		Sky B- B90/B70/B		Sky B- B90/B70/B			B-PR 70/B50
Zinc-Air battery size		31	12	13			
Rechargeable lit	hium-ion battery						•
Push button (Pro	ogram/Volume)	•	•	•		•	
Volume control				•		•	
Telecoil		•		•		•	
EasyPhone		•			•		•
Direct audio inp	ut				•		
Tamperproof bat	ttery door	•	•		•		
Indicator light		•	•		•	-	•
Phonak CROS B	compatibility	•	,		•		
Nano coating		•	•		•		•
IP rating		IPe	58 ¹	IPe	58 ¹	IP68 ¹	
Dimensions	LxWxD	29.5 x 13	x 7.4 mm	33.1 x 16.3	3 x 8.6 mm	35.9 x 15.9	9 x 8.7 mm
		(1.16 x 0.1 x 0.29")		(1.30 x 0.	1 x 0.34")	(1.41 x 0.63 x 0.34")	
	Weight	1.9 g (C).07 oz)	2.7 g (0.1 oz)		3.2 g (0.11oz)	
Rechargeable	No audio/media streaming					24	ł h
lithium-ion	5h of audio/media streaming					20) h
battery life ²	10h of audio/media streaming	-				16	5 h
Charging times ³	0% to 100%					3	h
	0% to 80%					1.5	5 h
	30 min fitting					10	min
		SlimTube HE	HE 10 680	SlimTube HE	HE 10 680	SlimTube HE	HE 10 680
Max. Power Out	put (dB SPL)						
	2 cc coupler	122	129	126	131	126	131
	Ear simulator	126	134	130	135	130	135
Max. gain (dB)	2 cc coupler	56	63	58	66	58	66
	Ear simulator	60	68	62	71	62	71
Frequency range	e – Ear simulator (Hz)	<100 - 6600	<700 - 6300	<100 - 5500	<100 - 6800	<100 - 5500	<100 - 5500
Working current	t – Ear simulator, HE 10 680 (mA)	1.2		1.2			
Fitting range SlimTube HE Regular tube		dB HL 0 10 20 30 40 50 60 70 80 90 100 1125 250 550 1k 2k 4k 8k Mild to moderately-severe		dB HL 0 10 20 30 40 50 60 70 100 110 1125 250 500 lik 2k 4k 8k Mild to severe hearing		d8 HL 0 10 20 30 40 50 60 70 100 110 1125 250 500 1k 2k 4k 8k	
		hearing loss, a configurations		loss, all audic configuration		loss, all audio configuration	

IP68 indicates that the hearing aid is water resistant and dust tight. It survived continuous immersion in 1 meter of water for 60 minutes and 8 hours in a dust chamber as per the IEC60529 standard. No traces of dust were evident within the housing.
 Actual battery life also depends on hearing loss, battery age and sound environment.
 Actual charge time depends on the remaining battery life, but it will not exceed 3 hours.


Product description

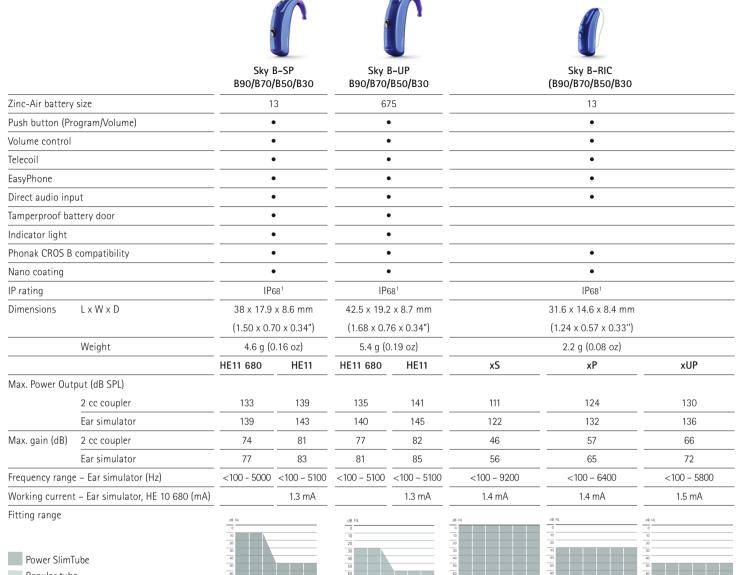



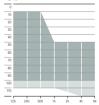

Mild to profound hearing loss, all audiometric configurations.

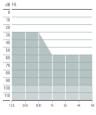

Mild to profound hearing loss, all audiometric configurations.



Mild to moderatelysevere hearing loss, all audiometric configurations.



Moderate to profound hearing loss, all audiometric configurations.

¹ IP68 indicates that the hearing aid is water resistant and dust tight. It survived continuous immersion in 1 meter of water for 60 minutes and 8 hours in a dust chamber as per the IEC60529 standard. No traces of dust were evident within the housing.
2 Actual battery life depends on hearing loss, battery age, and sound environment.

³ Actual charge time depends on the remaining battery life, but it will not exceed 3 hours.

Housing and hook colors

Housing



Acoustic couplings

Sky B-P and B-PR models			Colors
Sound outlet	Hook standard	Damped (HE10 680)	Transparent + 6 pediatric colors
		Undamped (HE10)	Transparent
	Hook tamperproof	Mini damped (HE10 680)	Transparent + 6 pediatric colors
Mini und		Mini undamped (HE10)	Transparent
	SlimTube HE	Left and right in four different lengths (0, 1, 2, 3)	
Earpiece	Dome	Open Dome in three sizes (S, M, L)	
		Closed Dome in three sizes (S, M, L)	
		Power Dome in three sizes (S, M, L)	
SlimTip	Hard material	Hollow shell	
		Solid shell	

Sky B-SP and B-UP models			Colors	
Sound outlet	Hook standard	Damped (HE11 680)	Transparent + 6 pediatric colors	
		Undamped (HE11)	Transparent	
Hook tamperproof		Mini damped (HE11 680)	Transparent + 6 pediatric colors	
		Mini undamped (HE11)	Transparent	
	Power SlimTube	Left and right in five different lengths (00, 0, 1, 2,3)		

Sky B-RIC models					
Receiver length	Available in four different lenghts (0,1,2,3) left and right				
Earpiece Dome Open Dome in three sizes (S, M, L)					
		Closed Dome in three sizes (S, M, L)			
		Power Dome in three sizes (S, M, L)			
SlimTip	Hard material	Hollow shell			
(available with extra canal lenght option)		Solid shell			
cShell	Hard material				

Wireless communication portfolio

All wireless accessories are compatible with all performance levels (B90/B70/B50/B30) and for all models.

Wireless accessory		Sky B-M	Sky B-P	Sky B-PR	Sky B-SP	Sky B-UP	Sky B-RIC
0020 0020 0030 0030 0030 0030 0030 0030	Phonak DECT II	•	•	•	•	•	•
•	Phonak PilotOne II	•	•	•	•	•	•
0	EasyCall II	•	•	•	•	•	•
	Phonak ComPilot II	•	•	•	•	•	•
	Phonak ComPilot Air II	•	•	•	•	•	•
	Phonak TVLink II	o 1	•1	o 1	• 1	•1	•1
	Phonak RemoteMic	•1	•1	•1	•1	•1	•1
¹ Requires Ph	onak ComPilot II or ComPilot Air II						
Roger™		Sky B-M	Sky B-P	Sky B-PR	Sky B-SP	Sky B-UP	Sky B-RIC
9	Roger 18		•		•		•
•	Roger 19					•	
* 5	Roger X / AS18		•		•		•
.	Roger X / AS19					•	
	Roger X / Phonak ComPilot II	•	•	•	•	•	•
	Roger MyLink	•	•	•	•	•	•

All Sky B models are compatible with Roger™ Touchscreen Mic, Roger™ Pass-around, Roger™ Multimedia Hub, Roger Select™, Roger Pen™, Roger™ EasyPen and Roger™ Clip-On Mic when using one of the above Roger receivers.

For information on Roger microphones please visit www.phonakpro.com/roger

Phonak Sky B-PR charging options



Phonak Charger Case BTE

The Phonak Charger Case is a charger, drying kit, and protective hard case all in one. Additionally, it includes a cleaning tool in the robust and splash-proof housing designed for two Sky B-PR hearing aids.



Phonak Power Pack

For charging Phonak Sky B-PR on the go, the Phonak Power Pack can be easily attached to the bottom of the Phonak Charger Case. It provides power for seven full charges of two Sky B-PR hearing aids.



Phonak Mini Charger BTE

The Phonak Mini Charger is a compact charger for two Sky B-PR hearing aids.

Key features

AutoSense Sky OS

AutoSense Sky OS is the only* dedicated pediatric operating system, specifically built for a child's listening environment. The new automatic program and feature selector in AutoSense Sky OS is up to 30% more precise for noiser classrooms and up to 39% more accurate in recognizing yelling and shouting as unwanted noise compared to the adult system.¹

* valid February 2018

SoundRecover2

SoundRecover2 improves high speech audibility by utilizing its adaptive frequency lowering algorithm that is only applied when the level of hearing loss requires it. Provides improved audibility of high-frequency information* while also maintaining the low and mid-frequency sounds, giving children access to a broader range of sounds.²

* compared with SoundRecover

Roger and directional setting

With Phonak Roger and directional setting, children using Sky B get the best of both worlds: superior speech understanding with the Roger system and automatically activated directional microphones on Sky B. The new improved setting will activate the directional microphone at lower noise levels, giving children access to their peers at softer conversational levels. In Phonak Target, the program 'Roger + Mic' offers a choice of three microphone modes; omnidirectional, Real Ear Sound and an automatically activated fixed directional.³

RogerReady

RogerReady makes adding a Roger receiver to a hearing aid simple and easy. Without the need for additional programming, devices with direct audio input and a Roger ear-level receiver automatically detect the signal from a Roger microphone and activate the 'Roger + Mic' program.

Indicator light

For added reassurance, an easy-to-understand indicator light visibly confirms the hearing aid is powered on, the Roger program is active and also warns of a low battery. All five Sky B BTE (RIC excluded) models include the indicator light within the transparent push button, to be activated by the fitter per need.

¹ Feilner, M., Rich, S., & Jones, C. (2016). Automatic and directional for kids – Scientific background and implementation of pediatric optimized automatic functions. Phonak Insight, retrieved from www.phonakpro.com/evidence

² Rehmann, J., Jha, S., & Allegro Baumann, S. (2016). SoundRecover² – the adaptive frequency compression algorithm. More audibility of high frequency sounds. Phonak Insight, retrieved from www.phonakpro.com/evidence

³ Jones, C., & Rakita, L. (2016). A powerful noise-fighting duo: Roger™ and Phonak directionality. Phonak Field Study News, retrieved from www.phonakpro.com/evidence

Fitting

Software	Phonak Target 5.3 or higher
Interfaces	iCube II, NOAHLink, HI-PRO, HI-PRO2

Performance levels

Programs	Premium (B90)	Advanced (B70)	Standard (B50)	Essential (B30)	
AutoSense Sky OS	Premium	Advanced	Standard	Essential	
Calm Situation	•	•	•	•	
Speech in Noise	•	•	•	•	
Comfort in Noise	•	•	•		
Music	•	•			
Speech in Loud Noise	•				
Speech in Car	•				
Comfort in Echo	•				
Additional programs					
Max. additional programs	5	4	3	2	
Speech in Wind	•				
Comfort in Echo	•				
Speech in Loud Noise	•	•			
Speech in 360°	•	•			
Speech in Noise	•	•	•	•	
Calm Situation	•	•	•	•	
Comfort in Noise	•	•	•	•	
Music	•	•	•	•	
Acoustic phone	•	•	•	•	
Custom program	•	•	•	•	
Streaming programs					
Max. streaming programs	4	4	4	4	
Bluetooth® audio + mic	•	•	•	•	
Bluetooth® phone / DECT + mic	•	•	•	•	
Roger / RemoteMic	•	•	•	•	
Audio jack	•	•	•	•	
<u> </u>					

Features	Premium (B90)	Advanced (B70)	Standard (B50)	Essential (B30)
RogerReady	•	•	•	
Roger and directional setting	•	•	•	
UltraZoom	Premium	Advanced	Standard	Essential
SNR-Boost	•	•	•	
FlexControl	•	•	•	
FlexVolume	•	•	•	
DuoPhone	•	•	•	
SoundRecover2	•	•	•	•
User Preference Tuning	•	•	•	•
Real Ear Sound	•	•	•	
Finetuning channels	20	16	12	8
WhistleBlock	•	•	•	•
NoiseBlock	•	•	•	•
WindBlock	•	•		
EchoBlock	•			
SoundRelax	•	•		
QuickSync	•	•	•	•
AOV	•	•	•	•
auto Acclimatization	•	•	•	•

Life is on

At Phonak, we believe that hearing well is essential to living life to the fullest. For more than 70 years, we have remained true to our mission by developing pioneering hearing solutions that change people's lives to thrive socially and emotionally. Life is on.

www.phonakpro.com





