

Phonak Sky™ Marvel

Hearing aid status and indicator light pattern

		Sky M-M and Sky M-SP	Sky M-PR
<input type="checkbox"/> Hearing aid is on	Repeating single blink		
<input type="checkbox"/> Hearing aid is on and Roger™ is accessible	Repeating double blink		
<input type="checkbox"/> Low battery	Continuous blink		
<input type="checkbox"/> Volume level change ***	Single blink per volume change		
<input type="checkbox"/> Program change ***	Single blink per program change		

* Low battery alert is initiated approx. 30 minutes before the battery needs to be changed.

** Low battery is initiated approx. 60 minutes before the battery needs to be charged.

*** Initiated either by a remote control or by the multi-function button.