




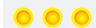

Phonak Sky™ V/B

Hearing aid status and indicator light pattern

Hearing aid status	Indicator light pattern
<input type="checkbox"/> Hearing aid is powered on	Repeating single blink 
<input type="checkbox"/> Hearing aid is powered on and Roger is accessible	Repeating double blink 
<input type="checkbox"/> Low battery *	Repeating triple blink 
<input type="checkbox"/> Volume level change **	Single blink per volume change 
<input type="checkbox"/> Program change **	Single blink per program change 

* Low battery is signaled approximately 30 minutes before the battery needs to be changed. It is recommended that you always have a new battery at hand.

** Either initiated by a remote control or by the push button.