

Phonak Fast Facts



Noahlink Wireless

What is it?

Noahlink Wireless is the new industry-standard programming device for Bluetooth® Low Energy (BLE) enabled hearing aids. It is a HIMSA manufactured device that wirelessly transfers fitting data from the computer to BLE enabled hearing aids – such as Phonak Audéo™ B-Direct hearing aids – completely cable-free.

Why use it?

Noahlink Wireless simplifies the fitting process:

- It is a universal programming device eliminating the need for proprietary programming devices
- It provides the flexibility to adjust settings of BLE enabled wireless hearing aids, completely cable free
- Unique to Phonak, Audéo B-Direct can be easily identified. Press the push button on the hearing aid to highlight it in the selection of available devices.



How is it used?

Noahlink Wireless is easy to use and to set up. Install Phonak Target 5.2 or higher and plug in the USB cable from the Noahlink Wireless into the fitting computer. Noahlink Wireless will be automatically installed and can be easily selected in the drop down menu of available fitting devices.

Open and close the battery door and click **[Connect]** to detect Phonak Audéo B-Direct. Devices in range will be shown. Press **[Play beep]** or the push button on the hearing aids to confirm which side/s you wish to assign to a client.

For more information on Phonak Audéo B-Direct, go to www.phonakpro.com/audeo-b-direct or contact your Phonak representative.

For more information on Noahlink Wireless, please refer to the FAQs on the HIMSA website <https://www.himsa.com/Products/NoahlinkWireless/ProductInformationSheets/tabid/4794/language/en-US/Default.aspx>