

Using Roger™ Touchscreen Mic and Roger receivers

Reference Sheet

roger

Roger Touchscreen Mic is simple and intuitive to use in the classroom and allows both teachers and students to easily know when the microphone is working and transmitting.

Microphone



Roger Touchscreen Mic

To switch on, press the On/Off button on the side of the microphone for 3 seconds. The green indicator lights will flash during the start up.

- Hang the Roger Touchscreen Mic around the speaker's neck with the lanyard and adjust height until the top of microphone is 8" (20 cm) from the mouth.
- If a student needs to be connected, position the Roger Touchscreen Mic within 4" (10 cm) of the Roger receiver and tap the **Connect** icon. Repeat with receiver on other ear.
- Roger Touchscreen Mic can be muted by pressing the Mute button. The indicator light around the button will now flash red.
- At the end of the day, switch Roger Touchscreen Mic off by pressing the On/Off button for 3 seconds. It will then ask you to confirm powering off. Tap the **yes check mark**. Charge daily with the correct USB charger or by placing in it the Roger Charging Rack.

Receiver



Roger X

- Place Roger X on the hearing aid using the appropriate audio shoe. It is now ready for use.
- Once receivers are connected to the Roger network, there is no need to reconnect them.
- If the receiver must remain at school, remove Roger X from the hearing instrument at the end of the school day as directed by your hearing care professional.



Roger design-integrated

- Design-integrated Roger receivers are custom designed for specific hearing aids and cochlear implants.
- Once receivers are connected to the Roger network, there is no need to reconnect them, even after turning them off and back on.
- Removal of the design-integrated receivers should not be necessary at school.