

# Using Roger™ Touchscreen Mic for small groups

Reference Sheet

roger

The new innovative Small Group mode utilizes a system of 3 intelligent embedded microphones working together in an advanced and adaptive way. When the Roger Touchscreen Mic is placed on a table between 2-5 students, the microphones will automatically orient to the student who is talking in the group.

## Small Group mode

- Unclip Roger Touchscreen Mic from the neck-worn lanyard.
- Place Roger Touchscreen Mic flat on the table in the center of the small group\*.
- Microphone will automatically transmit the voices close to it.
- When the small group activity has ended, clip the Roger Touchscreen Mic back on the lanyard for one speaker usage.

\* When the primary Roger Touchscreen Mic is used in Small Group mode, it will no longer transmit to a DigiMaster loudspeaker or be used in a MultiTalker Network.

