

Using Roger to create a small group network

Reference Sheet




If you need to talk to a small group of children while another teacher continues speaking over the main classroom network, you can easily create a temporary second network using a Roger feature called SubNet. No frequency planning is required, and both you and the smaller group's children can easily rejoin the main network (EndSub) when you're finished.

Set-up

Assign SubNet/EndSub function to Roger inspiro's left softkey

(This only needs to be done once):

1. Press **OK**.
2. Scroll down to **Settings**, Press **OK**.
3. If **Config softkeys** is highlighted, Press **OK**.
4. If **Left softkey** is highlighted, press **OK**.
5. Scroll to **SubNet/EndSub**, and press **OK**.
6. Press the  key to go back to main menu.

Use

Create SubNet

1. Press **SubNet** softkey. Left softkey will now show **EndSub**.
2. Hold Roger inspiro less than 10 cm (4 inches) from the receiver and press **Connect**.

EndSub

1. Press **EndSub** softkey.
2. Hold Roger inspiro less than 10 cm (4 inches) from the receiver and press **Connect**. Repeat for all receivers that were in the SubNet.

