

Using Roger Clip-On Mic


Reference Sheet

roger

Roger Clip-On Mic was designed as a simple to use microphone for a listener using Roger receivers. Using a Roger Clip-On Mic is as easy as pushing a button.

Daily use of Roger Clip-On Mic

1. Charge Roger Clip-On Mic either in the docking station or using the mini-USB power cord. Charge for 2 hours for maximum use.
2. Turn microphone on by pressing the  button for 2 seconds until status indicator turns white.
3. Place Roger Clip-On Mic on the talker's shirt using the in-built clip, or hanging on the lanyard (positioned within 8 inches/20 cm of the mouth).
4. Connect your Roger Clip-On Mic to a multimedia device in either of the following ways before switching on the multimedia device:
 - a) Connect the docking station to the audio device and dock the Roger Clip-On Mic.
 - b) Connect the micro-USB cable.
5. Charge the Roger Clip-On Mic daily after usage.

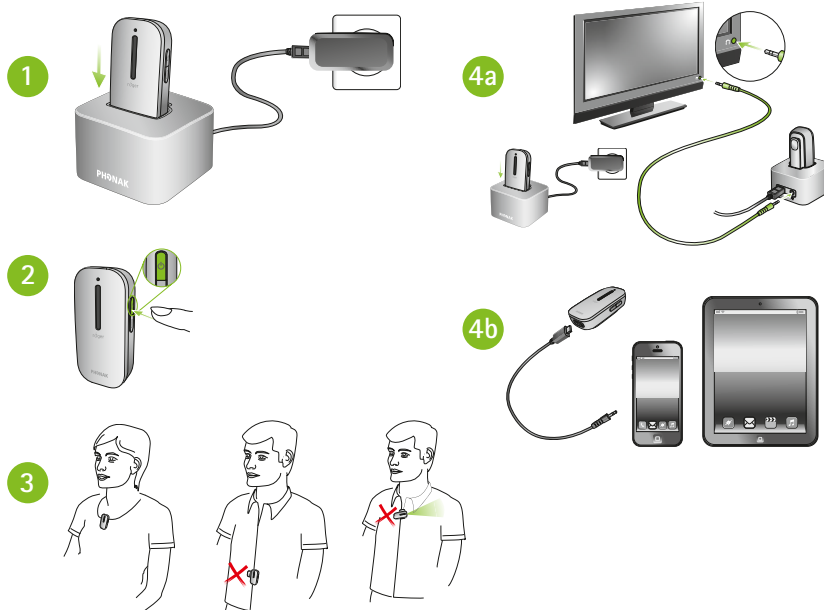

Roger Clip-On Mic indicator lights

- | | |
|--|-----------------------------------|
|  | Mute – Purple |
|  | Charging – Orange |
|  | Fully charged – Green |
|  | Battery low – Double blinking red |