

Phonak Fast Facts



Phonak ComPilot II

What is it?

The Phonak ComPilot II is the all-in-one streaming and remote control accessory for Phonak Venture and Phonak Belong™ hearing aids. It can receive speech and music from all Bluetooth enabled audio sources (cell phones, MP3 players, TV streamers, tablets, navigation systems, etc.) and stream these sounds wirelessly to both hearing aids in stereo sound quality, resulting in better understanding and more enjoyment. It can also connect to a Roger/FM receiver or 3.5 mm audio input. New in ComPilot II is the ability to simultaneously support two audio sources plus one TVLink or RemoteMic. The streaming time has also increased up to 24 hours. So clients can enjoy long streaming time to watch movies or play video games without any stress.



Why use it?

With ComPilot II, clients can boost the performance of the hearing aids. The directional microphone in ComPilot II picks up clear speech when speaking on the phone, even during calls in noisy situations. It also offers excellent stereo sound quality when watching TV or listening to music.

How to use it?

ComPilot II can stream audio immediately out of the box or it can be customized to the clients preferences by connecting to the Phonak Target 5.0 and newer fitting software. To use as a remote control it must be configured by the Phonak Target. For improved client usability, ComPilot II settings, including spoken messages and Bluetooth features, can be configured via the free Phonak RemoteControl App. The built-in sound demo makes ComPilot II extremely easy to instantly demonstrate in the office. Simply place the neckloop around your client's neck and press and hold the main button while turning it on. Your client will immediately hear the benefits ComPilot II offers.

For more detailed information visit www.phonakpro.com/compilot