

Phonak Fast Facts



Phonak ComPilot Air II

What is it?

The Phonak ComPilot Air II is the handy clip-on streamer and remote control accessory for Phonak Venture and Phonak Belong™ hearing aids. It can receive speech and music from all Bluetooth enabled audio sources (cell phones, MP3 players, TV streamers, tablets, navigation systems, etc.) and streams these sounds wirelessly to both hearing aids in stereo quality, resulting in better understanding and more enjoyment. The new integrated antenna makes ComPilot Air II smaller than ComPilot II without compromising the streaming stability. ComPilot Air II is ideal for clients who do not require more than four hours continuous streaming. For longer streaming time up to 24 hours, ComPilot II is a better solution.



Why use it?

Clients can really boost the performance of the hearing aids with ComPilot Air II. It offers clear speech understanding when using cell phones. It also has excellent stereo sound quality when watching TV (with TVLink II) or listening to music.

How to use it?

ComPilot Air II is ready out of the box to begin streaming audio. To use as a remote control it must be configured with Phonak Target 5.0 or newer. With only 3 buttons (main button, volume up and volume down) clients can easily use ComPilot Air II for effortless streaming and remote control functionalities. For clients who want more functionality like audio cable input or access to Roger/FM, ComPilot II is the ideal solution. ComPilot Air II settings, including spoken messages and Bluetooth features, can be configured via the Phonak RemoteControl App. The built-in sound demo makes ComPilot Air II extremely easy to instantly demonstrate in the office. Simply clip it on your client's clothing then press and hold the main button while turning it on. Your client will immediately hear the benefits ComPilot Air II offers.

For more detailed information visit: www.phonakpro.com/compilotair