




Technical Data

Phonak EasyCall II

Phonak EasyCall II connects Phonak wireless hearing aids to a cell phone. It is permanently attached and connected to the phone and streams the conversation directly to both hearing aids for unmatched sound quality and maximum speech understanding.

Description

- 1 Call button with light indicator
- 2 Micro-USB charging input
- 3 Microphone opening
- 4 Cell phone scratch protection ring
- 5 Optional adhesive tape

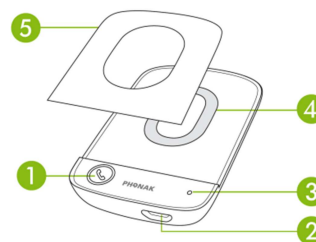

Feature overview

- Unpack and use: no fitting required
- Works with any Bluetooth enabled cell phone, even non-smartphones
- Attached onto the cell phone, never lost or forgotten
- Up to 10 hours talk time
- Up to 10 days standby time

Compatibility

- Use Phonak EasyCall for Spice and Quest wireless hearing aids
- Use Phonak EasyCall II for Venture wireless hearing aids

General information

Dimensions:	L x W x H: 69 x 45 x 12 mm (2.7 x 1.8 x 0.5 inches)
Weight:	24 g (0.71 oz.)
Color:	Black
Operating conditions:	The product has been designed for trouble-free operation without restrictions when used as intended, unless otherwise stated in the user guide 0° to 40° Celsius (32° to +104° Fahrenheit) and relative humidity of <85% (non condensing)
Transport and storage conditions:	-20° to 60° Celsius (-4° to +140° Fahrenheit) and relative humidity of <70% for a long period of time. Transport: <90% (non condensing)

Status light

Switching on:	Green for 2 seconds
Switching off:	Red for 2 seconds
Bluetooth pairing mode:	Rapid blue blink
During phone call:	Solid blue
Bluetooth connected:	Blink blue every 5 seconds for 1 minute
On and ready to connect:	Blink green every 5 seconds for 1 hour
Low battery warning:	13 short red blinks every minute 13 short red blinks after pressing the button when not in call (approximately 1 hour talk time left after first low battery indication)
Charging:	Solid red
Fully charged while charger attached:	Solid green when charging is complete Note: Charging state is not visible during phone call



Power and battery information

Battery:	Lithium Polymer 3.7 V Embedded, not removable Capacity: 320 mAh (typ.)
Operating time:	Up to 10 hours talk time Up to 10 days standby time
Charging information:	Input Voltage: Stabilized 5 Vd.c. +/- 5% Micro USB port Input current: max. 300 mA Charging time from empty battery: 2 h 45 min
Important notice:	For safety reasons, only use chargers supplied by Phonak or stabilized chargers with a rating of 5 Vd.c., min. 300 mA The battery cannot be over-charged. It may become warm during charging
Note:	Operating and charging time may depend on temperature and age of device.

Audio information

Audio input:	Speech, omni directional microphone
Connection to phone:	Bluetooth (speech only, no music). Wide band speech capable (HD voice) if supported by the phone/carrier.

Bluetooth

Bluetooth® profile:	Headset profile (HSP) Hands-free profile (HFP)
Frequency range:	2402.0 to 2480.0 MHz
RF power:	1 mW max.
Transmission range to cell phone:	Up to 1 m (3 ft.)
Bluetooth compliance:	Bluetooth version 3.0, Class 3
Max. paired devices:	8

Wireless link to hearing aids

Antenna type:	Ferrite antenna
Type:	Inductive ("HiBAN")
Operating frequency:	10.6 MHz
Modulation:	8 DPSK
Occupied bandwidth:	474.2 kHz
Field strength:	37.04dBµV/M@3m
Transmission range to hearing aid:	Up to 25 cm (10 ")

Standards

Europe:	EN 300 328 EN 300 330 EN 301 489 EN 60950-1 EN 62479
Canada:	RSS-210 Issue 8 RSS-102 Issue 4 RSS-GEN Issue 3
Japan:	ARIB-T66 (Bluetooth) EWPE (HiBAN)
USA:	FCC 47 CFR Part 15.209 (HiBAN) FCC 47 CFR part 15.247 (Bluetooth) FCC OET Bulletin No.65



The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Phonak is under license. Other trademarks and trade names are those of their respective owners.

Bluetooth pairing information

Press and hold the call button (2 seconds) to switch EasyCall II on or off.



If EasyCall II has never been paired, it will automatically be in pairing mode for 60 minutes after turning on. This is indicated by a rapidly blinking blue indicator light.

To manually start pairing mode, press and hold the call button for 10 seconds until the indicator light starts to rapidly flash blue.

On the phone, select "Phonak EasyCall II" from the list of Bluetooth devices and enter code "0000" if prompted.