

Phonak Fast Facts



Phonak EasyCall II

What is it?

The Phonak EasyCall II is a unique wireless accessory that connects any Bluetooth enabled cell or smart phone from all brands to Phonak Venture and Phonak Belong™ hearing aids. After picking up the phone, it automatically and wirelessly streams the call to both Phonak Venture and Phonak Belong hearing aids resulting in unmatched sound quality and maximum understanding.

Why use it?

With EasyCall II, clients can benefit from improved speech intelligibility on their cell phone with a simple click of a button¹.

Clients do not have to worry about the phone brand. EasyCall II works with any Bluetooth enabled phone, even with non-smartphones.



How to use it?

EasyCall II is easy to use. It works out-of-the-box and requires no fitting. Once attached to the phone, it is always ready to use. Pairing to the phone is performed only once. Switch EasyCall II on and it will immediately be in pairing mode. In the Bluetooth settings of the phone, select and connect EasyCall II and you are done. There is nothing left to do but push the EasyCall II button and answer the call.

For more detailed information visit www.phonakpro.com/easycall

¹ Appleton-Huber J, (2013) 'Phonak DECT CP1 cordless phone' Field Study News. Phonak AG: 2013