

Phonak Fast Facts




Phonak RemoteControl App

What is it?

The Phonak RemoteControl App turns any smartphone (Android and iOS) into a remote control. It provides clients with Phonak Venture and Phonak Belong™ hearing aids the opportunity to have more control and flexibility through the use of their smartphone.

Why use it?

The RemoteControl App offers many functionalities that clients can control. For example, they can independently control the volume of the left or right hearing aid or change the language setting of VoiceAlerts. It is also possible to control which external Bluetooth sources they want to stream audio from. They can also directly select their hearing programs or change the direction of ZoomControl (front, back, left or right) giving clients the ability to choose in which direction to focus their hearing and much more.

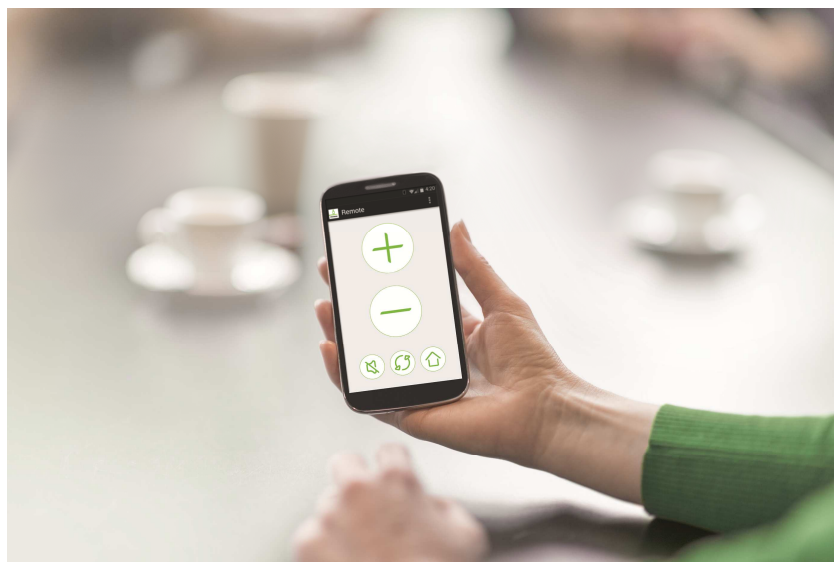

How to use it?

The RemoteControl App requires a Phonak ComPilot II or ComPilot Air II streamer as an interface to the Phonak Venture and Phonak Belong hearing aids. While the streamer can be used directly for simple volume and program adjustments, the RemoteControl App extends the control options.

The RemoteControl App can be downloaded from the Apple App Store or Google Play. Once installed, the easy-to-follow pairing wizard will help to connect the app to the streamer. A demonstration mode is also available to assist clients in getting started. Required are devices with Apple iOS 7 or higher and Android 4.0 or higher.

For more detailed information visit www.phonakpro.com/rc-app