

Phonak

Fast Facts

Roger™ installation via Roger iN microphones

What are Roger iN microphones?

Roger iN microphones are three new Roger microphones: Roger Select iN, Roger Pen iN and Roger Table Mic II iN, designed to be used with the Marvel hearing aids. They have the capability to install two Roger receivers into a pair of Marvel hearing aids in order to directly stream from a Roger microphone without physically attaching an external receiver.


Why use it?

After a simple installation process, clients can use a Roger iN microphone with their Marvel hearing aids to enhance hearing performance in noise and over distance. This discretely designed solution enables clients to fully participate in group conversations even in very noisy environments such as restaurants, meetings at work and school activities.


How to use it?

To begin, turn on the Roger iN microphone and the Marvel hearing aid. Using the pin tool, press and hold the installation button for two seconds and place the Roger iN microphone near the hearing aid. The installation process is fully automatic and a series of indicator lights will confirm that the receiver has been installed. For those who may need to restore the receiver for use with a different hearing instrument, the same process is used to uninstall the receiver.

Pin tool and installation button
Roger Select iN

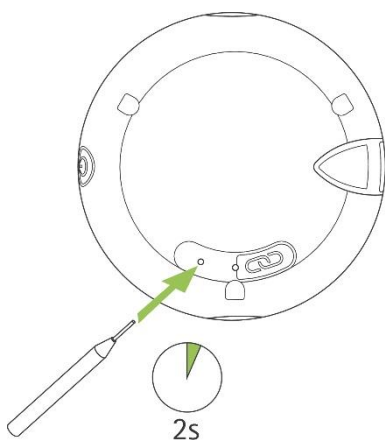

Pin tool and installation button
Roger Pen iN

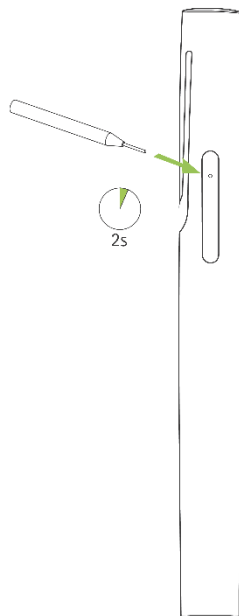

Pin tool and installation button
Roger Table Mic II iN

