


Phonak

Roger Select TV guide


Analog cable plugged into headphone output

Plug the supplied analog cable into the TV's headphone or audio output. This may mute the loudspeakers of the TV. Some TVs offer the possibility to enable headphone and loudspeaker in parallel. Please check the audio settings of your TV. Alternatively, you may also find an audio output on your TV's set-top box or streaming box.

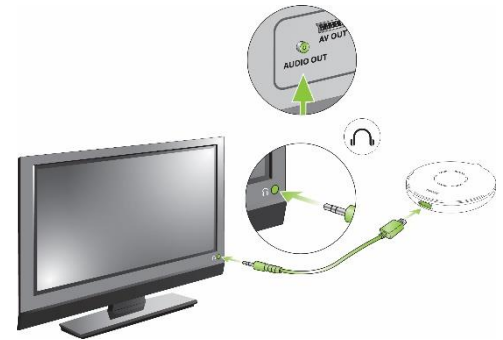

Roger Select placed near loudspeaker


Place the Roger Select near the TV's loudspeaker. On flat screens, the loudspeakers are usually on the back side, so you may place the Roger Select behind your TV.

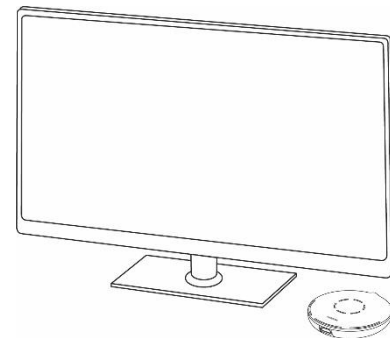

Analog cable with Cinch adapter

Required parts: Cinch adapter cable

Plug the cinch cable into the audio output of your TV. Plug the audio cable into the other end of the cinch cable and your Roger Select.

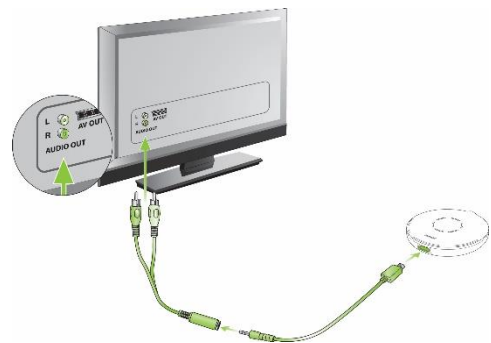

Digital cable with digital to analog converter

Required parts: Digital to analog converter

Plug the optical cable into the digital audio output of your TV. Plug the other end of the optical cable into the DA converter. Then plug the supplied audio cable in the audio output of your DA converter and plug the other end of the cable into Roger Select. Switch on the TV, Roger Select and the DA Converter.

