

Phonak

Guide/ FAQ

Using Roger™ Touchscreen Mic and automatic Primary handover

The automatic Primary handover feature allows to keep automatically the Roger network intact when the Roger Touchscreen Mic, which is the Primary of the Roger Network is leaving.

Why use it?

For modern teaching, a Roger Network supports MTN – MultiTalker Network. A MTN allows up to 35 Roger microphones such as Roger Touchscreen Mic or Roger Pass-around to be used in the same MTN Roger network. A MTN Roger Network can be created by any Roger Touchscreen Mic, which will become the Primary for the MTN. However, for some reasons the Primary might leave the room. The replacement of the Primary leaving the room by another Roger Touchscreen Mic available in the network will be done seamlessly and automatically.

How to use it?

Simply, use any Roger Touchscreen Mic running firmware v2.0 or higher to create a MTN Roger Network and to be a Primary. All Roger Touchscreen Mic in the MTN Roger network shall run firmware v2.0 or higher to support the automatic Primary handover.

