

# Phonak

## Fast Fact

### Phonak CeruShield Disk

#### What is it?

Phonak CeruShield™ Disk is the new wax system from Phonak designed as an easy to use dispenser wheel.

The CeruShield Disk consists of :

- 8 new CeruShield filters
- 8 positions for easy disposal of used wax filters

CeruShield filters protects the receiver from wax related problems and offers clients longer wearing time between filter changes. CeruShield filters have been designed specifically to be used with the SDS 4.0 receivers



#### Why use it?

Benefits for you:

- Reduced maintenance concerns

Benefits for your clients

- Longer wearing time between wax filter changes
- Less time spent on device maintenance
- Easy handling

#### How to use it?


RIC hearing aids fit with domes

- CeruShield filter for the receiver

RIC hearing aids fit with custom ear pieces have double wax protection:

- CeruStop filter for the custom tip
- CeruShield filter for the receiver

Please note that Phonak CeruShield is only compatible with Phonak SDS 4.0 Receivers and is NOT compatible with older receivers.

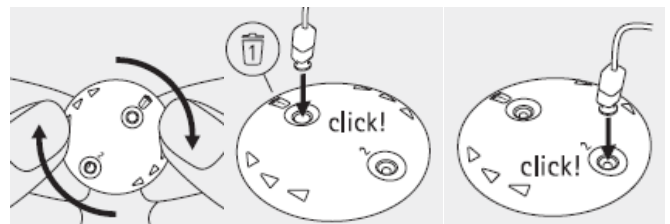

For more information on the Phonak Sound Delivery System, go to [www.phonakpro.com](http://www.phonakpro.com) or contact your Phonak representative.