

Roger Upgrader インストールガイド



ver.1.16

インストール手順

1. Roger Upgrader v1.16
の【ZIP】をクリックしてPCに
保存します。

Roger™ Upgrader

Roger Upgrader is designed to ensure the Roger compatibility of Phonak devices, such as: wireless microphones (Roger Touchscreen Mic, Pass-around, Inspiro, DynaMic); loudspeakers (Roger Dynamic SoundField); and multimedia accessories (Roger Multimedia Hub, AudioHub).

	Guide: Roger Upgrader fitting guide for schools	Share	PDF 0.4 MB
	Software Download: Roger Upgrader v1.16 for schools	Share	ZIP 90.7 MB

Click here

2. フォルダ内の
【setup.exe】をクリックし
て、インストールウィザードを
開始します。

名前

- Documents
- ISetupPrerequisites
- 1031.mst
- 1033.mst
- 1034.mst
- 1036.mst
- Autorun.inf
- Data1.cab
- Roger Upgrader
- setup.exe

Click here

3. インストール
ウィザードが開始されま
すので、画面の指示に従
ってお進みください。

Roger Upgrader 1.16

Welcome to the InstallShield Wizard for Roger Upgrader 1.16

The InstallShield(R) Wizard will install Roger Upgrader 1.16 on your computer. To continue, click Next.

We recommend to disable Antivirus software for the time of the installation.

To continue, click Next.

BuildNo. 2673

< Back Next > Cancel

Click Here

Roger Upgrader 1.16

PHONAK License Agreement

Please read the following license agreement carefully.

END USER LICENSE AGREEMENT

IMPORTANT
This License Agreement sets out important and strict Terms and Conditions which limit your use of this Software.
READ THIS AGREEMENT CAREFULLY AND COMPLETELY BEFORE USING THE SOFTWARE.
You must use the scrolling feature to view the entire License Agreement.
YOU MUST AGREE IN FULL WITH THE TERMS OF THE LIMITED LICENSE OFFERED IN THIS LICENSE AGREEMENT BEFORE USING THIS SOFTWARE.

TERMS AND CONDITIONS

Click 'Accept' to accept the license agreement and to continue the installation or click 'Cancel' to cancel the installation.

InstallShield

< Back Accept Cancel

Click Here

Roger Upgrader 1.16

PHONAK User Settings

Please select your settings and press 'Next' to continue.

Install Roger Upgrader 1.16 to:
C:\Program Files (x86)\Phonak\Roger Upgrader\ Change...

InstallShield

< Back Next > Cancel

Click Here

Roger Upgrader 1.16

PHONAK Ready to Install the Program

The wizard is ready to begin installation.

Click Install to begin the installation.

If you want to review or change any of your installation settings, click Back. Click Cancel to exit the wizard.

InstallShield

< Back Install Cancel

Click Here

4. 左の画面が表示され
ればインストール完了
です。
【Finish】をクリックして、
終了となります。

Roger Upgrader 1.16

InstallShield Wizard Completed

The InstallShield Wizard has successfully installed Roger Upgrader 1.16. Click Finish to exit the wizard.

☐ Show the Windows Installer log

< Back Finish Cancel

Finish