

노아링크 와이어리스 버전 업그레이드 및 다운그레이드

Target 피팅 프로그램에서 보청기 인식 시, '피팅 장비 펌웨어 오류'가 발생하거나(1)

HIMSA에서 피팅 장비 업그레이드 후, 문제가 발생한 경우(2) 아래 방법에 따라 문제를 해결하시기 바랍니다.



1. 버전 업그레이드 방법




◀ 피팅 장비 펌웨어 오류 메시지



[해결 방법]

① PC에 노아링크 와이어리스를 연결합니다.


② Target - [Trial & 도구] - [펌웨어 업데이트] - [Noahlink Wireless...] 를 클릭합니다.

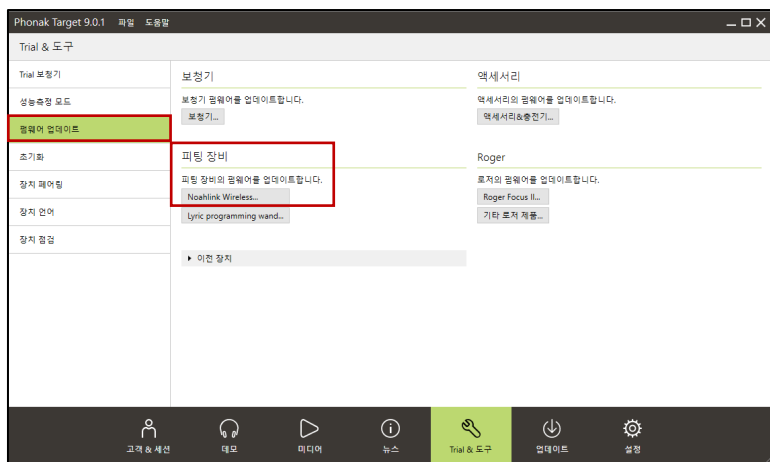

③ [Upgrade]를 클릭하여 업그레이드를 진행합니다.

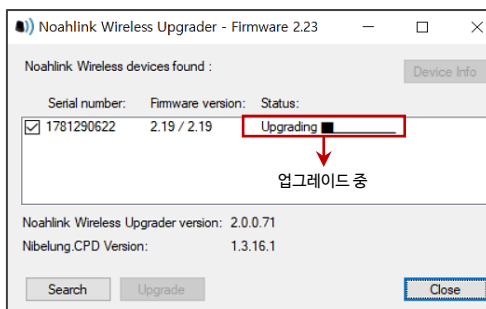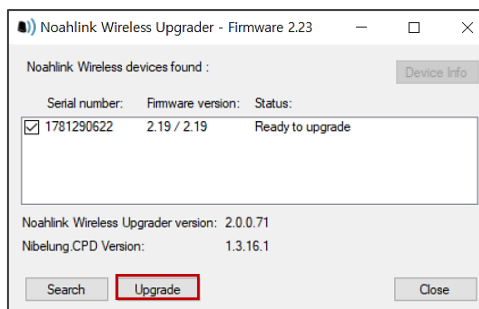


④ 업그레이드 완료 후, 노아링크 와이어리스 버전을 확인합니다.

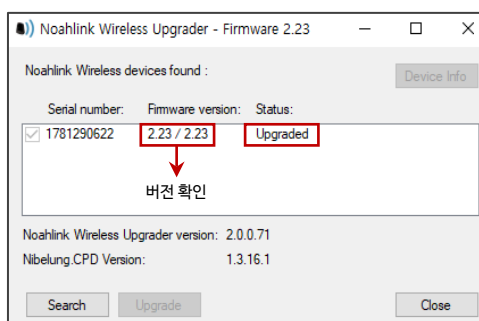

2. 버전 다운그레이드 방법

- ① PC에 노아링크 와이어리스를 연결합니다.
- ② HIMSA 홈페이지(www.himsa.com)에 접속하여 [Download] - [Noahlink Wireless Downloads]를 클릭합니다.
- ③ 맨 하단에 있는 [Noahlink Wireless Firmware Upgrader] 다운로드 링크를 클릭하여 설치합니다.

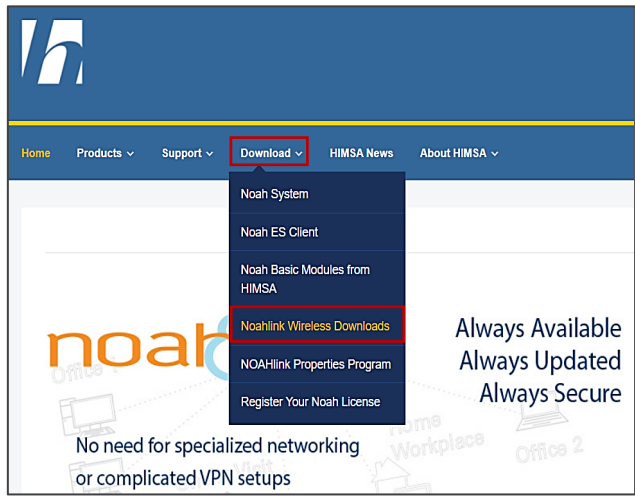

Always Available
Always Updated
Always Secure

No need for specialized networking
or complicated VPN setups


WARNING: Do not unplug your Noahlink Wireless unit during the upgrade process! If you do so, the unit being upgraded will no longer work. The upgrade process may take 2-3 minutes.

6. When the upgrade is complete, the status field for the unit(s) will change to "Upgraded".

Important: HIMSA has recently implemented a new security certificate for our downloads. Virus protection systems may not yet recognize it as safe and you may receive a security warning from your software. If you receive a security warning, it is safe to proceed with the installation.

| Patch/Update | Description | Size | Download |
|-------------------------------------|--|--------|----------------------|
| Noahlink Wireless Firmware Upgrader | Includes firmware v. 2.24 November 2022 | 9.9 MB | Link |

- ④ 키보드에 있는 [Ctrl] + [Alt] + [Shift] + [D]키를 동시에 누르면 [Downgrade]가 생성됩니다.

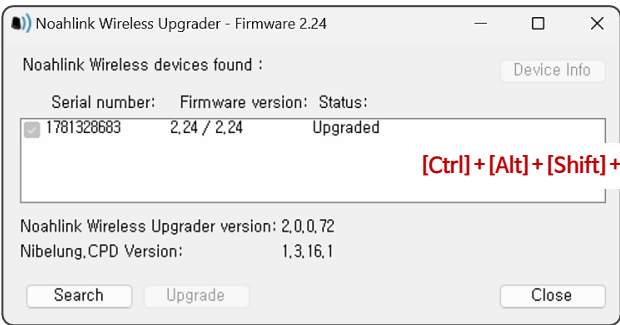


Noahlink Wireless devices found :

| Serial number: | Firmware version: | Status: |
|----------------|-------------------|----------|
| 1781326683 | 2.24 / 2.24 | Upgraded |

Noahlink Wireless Upgrader version: 2.0.0.72
Nibelung.CPD Version: 1.3.16.1

Search Upgrade Close

[Ctrl] + [Alt] + [Shift] + [D]

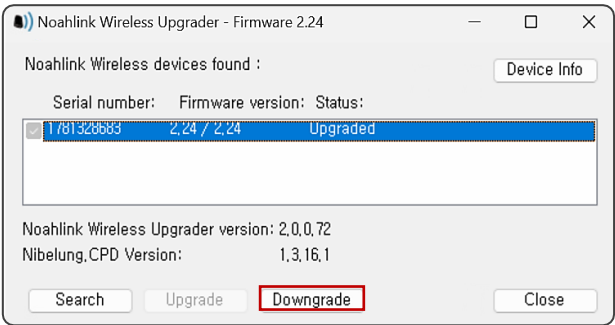


Noahlink Wireless devices found :

| Serial number: | Firmware version: | Status: |
|----------------|-------------------|----------|
| 1781326683 | 2.24 / 2.24 | Upgraded |

Noahlink Wireless Upgrader version: 2.0.0.72
Nibelung.CPD Version: 1.3.16.1

Search Upgrade Downgrade Close

- ⑤ 다운그레이드 버전을 확인한 후, 키보드에서 숫자 ‘1’키를 누릅니다(현재 연결된 노아링크 와이어리스).
그 다음, 키보드에서 알파벳 ‘Y’를 눌러 다운그레이드를 진행합니다.

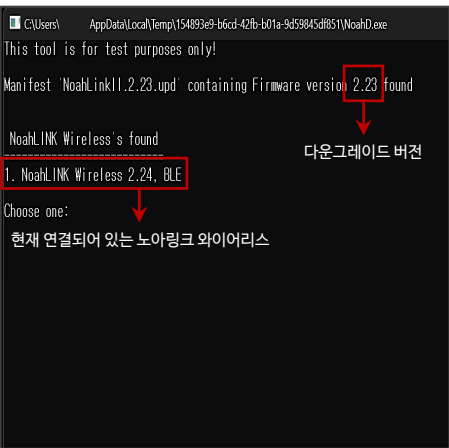


Manifest 'NoahLinkII.2.23.upd' containing Firmware version 2.23 found

NoahLINK Wireless s found

1. NoahLINK Wireless 2.24, BLE

Choose one:

현재 연결되어 있는 노아링크 와이어리스

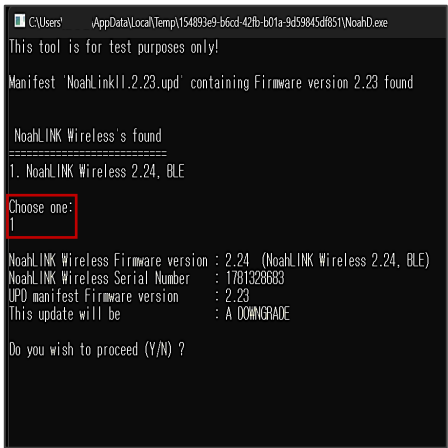

Manifest 'NoahLinkII.2.23.upd' containing Firmware version 2.23 found

NoahLINK Wireless s found


1. NoahLINK Wireless 2.24, BLE

Choose one:

1

NoahLINK Wireless Firmware version : 2.24 (NoahLINK Wireless 2.24, BLE)
NoahLINK Wireless Serial Number : 1781326683
UPD manifest Firmware version : 2.23
This update will be : A DOWNGRADE

Do you wish to proceed (Y/N) ?

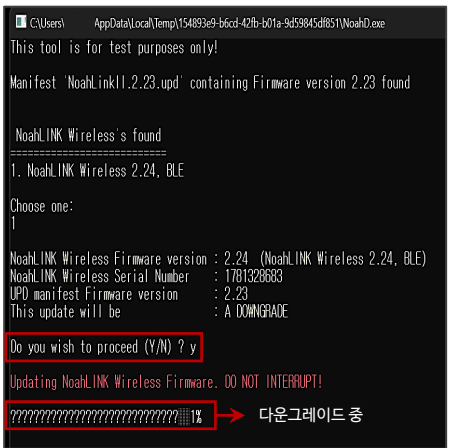

Manifest 'NoahLinkII.2.23.upd' containing Firmware version 2.23 found

NoahLINK Wireless s found

1. NoahLINK Wireless 2.24, BLE

Choose one:

1

NoahLINK Wireless Firmware version : 2.24 (NoahLINK Wireless 2.24, BLE)
NoahLINK Wireless Serial Number : 1781326683
UPD manifest Firmware version : 2.23
This update will be : A DOWNGRADE

Do you wish to proceed (Y/N) ? y

Updating NoahLINK Wireless Firmware. DO NOT INTERRUPT!

Progress bar: 1% → 다운그레이드 중

- ⑥ 다운그레이드 완료