Fitting Guide

roger

Nov 23, 2021

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Manual firmware change

USB devices (i.e. microphones/ AudioHub etc.)

Use this assistant to ensure the compatibility of Phonak USB devices such as wireless microphones, loudspeakers and multimedia accessories.

Start assistant

1 Roger 2 Connection 3 Selection 4 Programming 5 Finish

Please connect all your USB devices one after the other.

Update running...

This process may take several minutes. Please do not disconnect the active device or exit this software.

Do not disconnect the active device!
Do not exit this software until the end of the process!

Identified devices

Device	Serial Number	New version	Status
Roger Touchscre...	4711_0815	2.0.41947	94%

Continue

1 Roger 2 Connection 3 Selection 4 Programming 5 Finish

Please connect all your USB devices one after the other.

Update running...

This process may take several minutes. Please do not disconnect the active device or exit this software.

Do not disconnect the active device!
Do not exit this software until the end of the process!

Identified devices

Device	Serial Number	New version	Status
Roger Touchscre...	4711_0815	2.0.41947	10%

Continue

1 Roger 2 Connection 3 Selection 4 Programming 5 Finish

Please connect all your USB devices one after the other.

All devices are upgraded.

Do not disconnect the active device!
Do not exit this software until the end of the process!

Identified devices

Device	Serial Number	New version	Status
Roger Touchscre...	4711_0815	2.0.41947	Upgraded

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