

Phonak Fast Facts



Phonak EasyCall

What is it?

The Phonak EasyCall is a unique wireless accessory that connects any brand of Bluetooth-enabled cell phone to Phonak wireless hearing aids. After picking up the phone, it automatically and wirelessly streams the call to both hearing aids resulting in unmatched sound quality and maximum understanding.

Why use it?

With EasyCall, patients can benefit from improved speech intelligibility on their cell phone with a simple click of a button¹.

You don't have to worry about what brand of cell phone your patient has. EasyCall works with any Bluetooth-enabled phone, even with non-smartphones.

How to use it?

EasyCall is easy to use. It works out-of-the-box and requires no setup in Phonak Target™. Simply pair it to the phone.

Pairing to the phone is performed only once. Switch EasyCall on and it will immediately be in pairing mode. In the Bluetooth settings of the phone, select and connect EasyCall and you are done. Attach EasyCall to the cell phone, and push the button to answer the call.

EasyCall is compatible with Spice+ and Quest hearing instruments.

Easy Call II is compatible with Venture hearing instruments.



¹ Appleton-Huber J, (2013) 'Phonak DECT CP1 cordless phone' Field Study News. Phonak AG: 2013

For more detailed information visit:
www.phonakpro-us.com/easycall

PHONAK
life is on